Rollator Series H1170

applies to H1170 - H1179

User instructions
Bedienungsanleitung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso
Bruksanvisningar
Gebruiksaanwijzing
Instrukcja obsługi

User instructions

3		User instructions
22		Bedienungsanleitung
40		Mode d'emploi
58		Instrucciones de uso
76		Istruzioni per l'uso
94		Bruksanvisningar
97		Gebruiksaanwijzing
100		Instrukcja obsługi

Rollator Series H1170

English User instructions

Intended use

This rollator is intended to be used by persons with reduced mobility as a walking aid, as a wheelchair and for transporting smaller objects. This rollator model is designed for indoor and outdoor use. It is intended for use on solid ground by persons with a maximum weight of 300 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

Before using your rollator

Read the user instructions carefully before assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

You can assemble this rollator on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor. The rollator can be used by different people. Before use, check that the height is set correctly.

Important: Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.

If you have any questions regarding the assembly, the electric wheelchair or if you are unsure about anything, please contact our customer service at the e-mail address: **support@helavo-mobility.com**

Scope of Delivery:

The Helavo Series H1170 comes with a frame and the following parts:

- Removable cup holder
- Removable cane holder
- Basket
- Wing screws
- Front wheels
- Rear axle with rear wheels
- Legrests
- Velcro straps
- User instructions
- Product insert
- Step aid

Description of the rollator

H1170 Series Rollators:

Helavo model series H1170 consists of the following rollators:

- H1170 Hybrid-Rollator, green

Symbols


Manufacturer

CE-Marking

Date of manufacture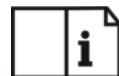

Instructions for use

Catalog No.

Maximum load capacity

Batch code

Unique device identifier

Medical Device

Safety

Warning: The rollator can provide assistance to a person with a maximum weight of 300 lbs. The maximum load of the basket of 11 lbs should not exceeded.

Follow the safety instructions

1. Make sure the parking brake ("L") is active when the rollator is parked.
2. Check the handle height and make sure the height adjustment ("N") is locked before use.
3. Ensure that the legrests are correctly attached to the mechanism and secured in place.
4. Make sure the walker is completely unfolded.
5. Keep your hands away from the folding mechanism to avoid entrapment.

What not to do

1. Do not sit on the rollator if the parking brake ("L") is not active or if it is at an angle.
2. Do not drop onto the seat ("E").
3. Do not hang any objects on the brake handles ("L") or hand grips ("K").
4. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the electric wheelchair or if you are unsure about anything, please contact our customer service at the e-mail address: **support@helavo-mobility.com**

Assembly instructions:

1. Remove the rollator from the packaging and remove the protective foils. Spread out the parts for assembly as shown.

2. Remove the wing screw ("O") on the rollator frame with the marking "RB".

3. Insert the rear wheel axle frame with rear wheel ("Q") marked "RB" into the frame at the corresponding "RB" marking. Both markings should face the outside of the rollator. Make sure the wing-screw ("O") holes align with those on the frame.

4. Fasten the rear wheel axle frame ("Q") with the wing screw ("O") you removed. The grey wing-screw head points inward.


Assembly instructions:


5. Now repeat steps 2–4 for the second rear wheel axle frame ("Q") marked "LB" on the other side of the frame.

6. Remove the wing screw ("F") on the rollator frame with the marking "RF".

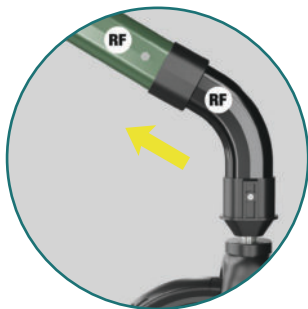
7. Insert the front wheel frame with front wheel ("J") marked "RF" into the frame at the corresponding "RF" marking. Both markings should face the outside of the rollator. Make sure the wing-screw ("F") holes align with those on the frame.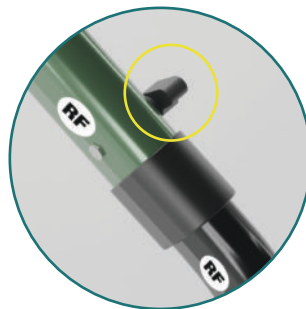

8. Fasten the front wheel frame with the wing screw ("F") you removed. The grey wing-screw head points inward.


Assembly instructions:

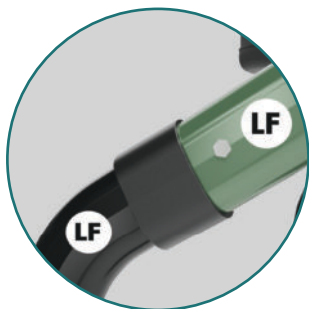9. Now repeat steps 6–8 for the second front wheel ("J") frame marked "LF" on the other side of the frame.

10. Put the rollator in an upright position and unfold it. **IMPORTANT:** Make sure the rollator is completely unfolded.

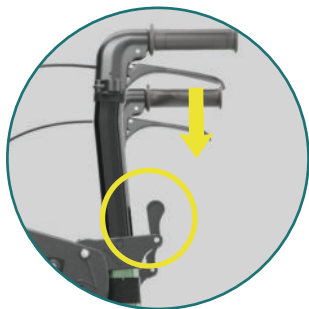
11. Insert the handles ("K") one after the other into the frame. The height-adjustment ("N") lever must be open and pointing upward when inserting; press it down to lock the height.

12. Fold the armrests ("C") up on both sides and secure them by inserting the bolt. Make sure the armrest holes align (are flush) with those on the frame.

Assembly instructions:

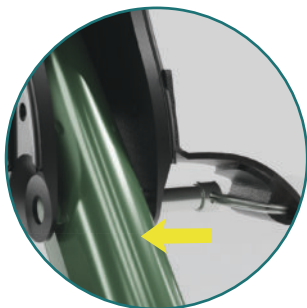13. Do this on both sides of the walker. **IMPORTANT:** Make sure the bolt is pushed all the way through to the end.

14. Insert the legrest ("H") marked "RF" into the legrest attachment ("G") by pressing it down onto the mechanism.

15. Repeat this step for the second legrest ("H") marked "LF" on the other side of the frame.

16. Our recommendation: Secure the brake cables on both sides of the frame using the velcro tape as shown.

Adjusting the height

Lift the height adjustment lever ("N") on the rollator frame to release the handle ("K") and adjust it to the desired height.

Make sure to press the height adjustment lever ("N") down again so the handle ("K") is securely locked in place. Repeat the process on the other side.

Ensure that both handles ("K") are set to the same height.

Handle height: 80 - 92,5cm / 31.49 - 36.41 inch

Your ideal handle height


Adjust the handles ("K") of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle, your elbow should be slightly bent. If you find that you need to bend over or keep your arms bent at a right angle, adjust the height of the handles.

Brakes

1. Pull the brake handles ("L") up to brake the rollator.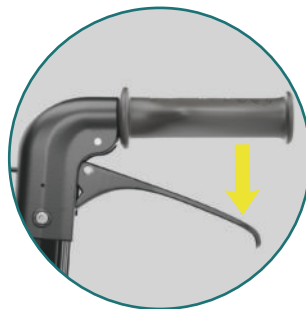


2. Push the brake handles ("L") down to lock the rear wheels. In this position, the parking brake of the rollator is active.

To release the parking brake again, pull the brake handles ("L") upward.

Adjusting the brakes


To adjust the rear brake "S", adjust the adjustment screw and locking nut. These are located on the rear wheels ("Q") of the rollator. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.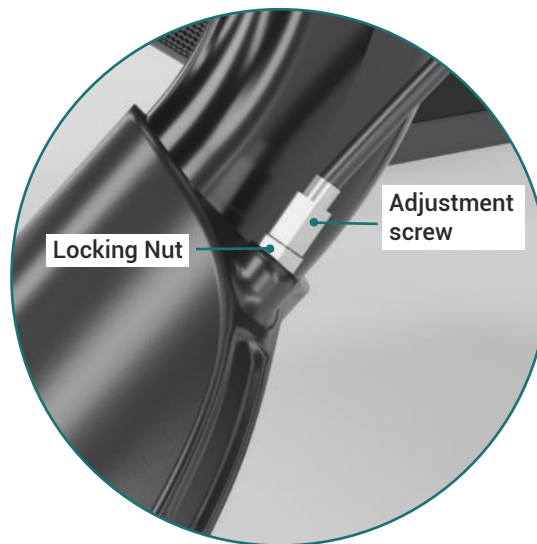

Important

If the brake adjustment has been made so that the rear brake ("S") is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

Removable cane holder

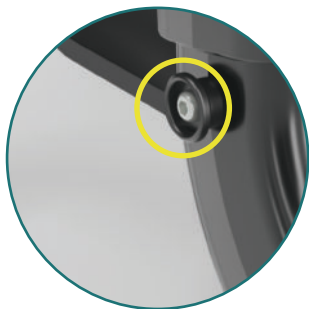1. Attach the cane holder ("P") to the side of the frame in the provided position.

2. To do this, slide the cane holder ("P") onto the holder from above.

Removable cup holder

1. Attach the cup holder ("A") to the side of the handle in the provided position.


2. To do this, slide the cup holder ("A") onto the holder from above.

Folding and storing the rollator

1. Remove the legrests("H"). You can store them in the basket.

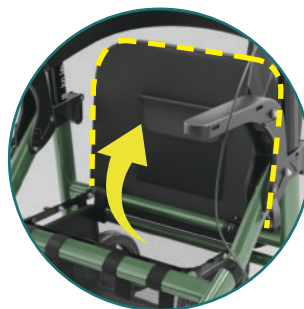2. Lift the seat ("E") of the rollator.

3. Pull up the black metal brace inside the basket to fold the rollator.

Conversion of the rollator

When using the rollator in its conventional function as a rollator, make sure that:

1. The backrest ("B") faces the front wheels.
2. The legrests ("H") are removed or attached to the rear of the rollator in the center.

Converting the rollator for use as a wheelchair

1. Fold the backrest ("B") to the other side, facing the rear wheels.
2. Lift the legrests ("H") and place them on their side facing forward on the leg rest attachment part. Push them all the way down so that they are securely attached.
3. Fold down the footrests ("I") one after the other after you have sat down on the seat of the wheelchair ("E") .

Important

Make sure you sit down safely on the seat ("E") and also stand up safely again. First sit down and then fold the footrests ("I") down. Fold the footrests up first and then stand up.


Rollator

Wheelchair

Maintenance

1. Check the rollator at regular intervals for loose wing and locking screws ("F"/"O") and defective height adjustments ("N").
2. Periodically check for proper operation of the rear wheel brakes ("S").
3. Replace damaged or worn parts.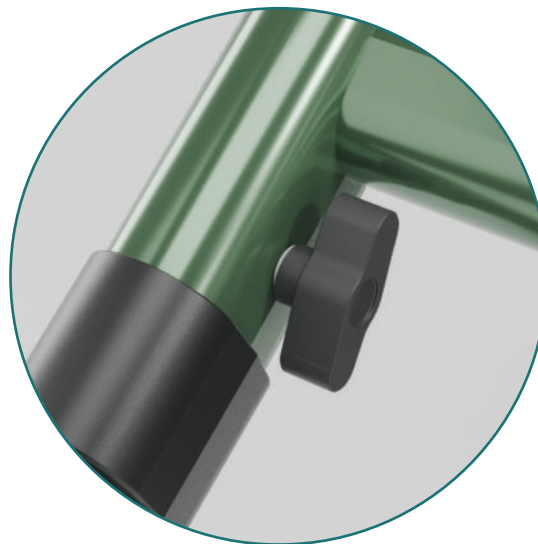

Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws or wheels ("F"/"O"/"J"/"Q").
2. Do not use the rollator if the wheels ("J"/"Q") are defective.
3. Do not use the rollator if the height adjustment ("N") of the handles does not engage properly or is defective.
4. Do not use the rollator if the brake ("L"/"S") does not work properly.
5. Do not use the rollator if it no longer engages properly when unfolded.

Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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