Rollator Walker H1320

User instructions

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Mode d'emploi

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Bruksanvisningar

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User Instructions

4		USER INSTRUCTIONS
19		BEDIENUNGSANLEITUNG
34		MODE D'EMPLOI
49		INSTRUCCIONES DE USO
64		ISTRUZIONI PER L'USO
79		BRUKSANVISNINGAR
82		GEBRUIKSAANWIJZING
85		INSTRUKCJA OBSŁUGI

Rollator Series H1320

English User instructions

Intended use

This rollator is intended to be used by persons with reduced mobility as a walking aid and for transporting smaller objects. This rollator model is designed for indoor and outdoor use. It is intended for use on solid ground by persons with a maximum weight of 300 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

Before using your rollator

Read the user instructions carefully **before** assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

You can assemble this rollator on your own in a few steps. Assembly takes about 3 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor.

The rollator can be used by different people. Before use, check that the height is set correctly.

Important: Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Scope of delivery

The Helavo Series H1320 comes with a frame and the following parts:

- Removable cane holder
- Backrest
- Bag
- Main Frame with front and rear Wheels and Seat
- User instructions
- Product insert
- Step aid

Description of the rollator

H1320-Series rollators

Helavo model series H1320 consists of the following rollators:

- H1320: AllroundRollator, black

Symbols

Manufacturer

Date of manufacture

Catalog No.

Batch code

Medical Device

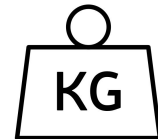
Unique device identifier

Instruction for use

CE-Marking

Maximum load capacity

Safety

Warning: The rollator can provide assistance to a person with a maximum weight of 300 lbs. The maximum load of the bag of 11 lbs should not exceeded.

Follow the safety instructions

1. Make sure the parking brake ("G") is active when the rollator is parked.
2. Check the handle height and make sure the height adjustment ("I") is locked before use.
3. Make sure the rollator is completely unfolded and locked into position with a "click" sound.
4. Keep your hands away from the folding mechanism to avoid entrapment.

What not to do

1. Do not use the rollator as a wheelchair.
2. Do not sit on the rollator if the parking brake ("G") is not active or if it is at an angle.
3. Do not "drop" onto the seat ("C").
4. Do not hang any objects on the brake handles ("G") or handlegrips ("A").
5. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Assembly instructions

1. Remove the rollator from the packaging and remove the packaging material.

2. Put the rollator in an upright position and pull it apart by both handles ("A"). Push it down on both sides of the seat ("C") as shown. **Important: For the rollator to be fully unfolded, it must lock into place with a "click" sound.**

3. Loosen and remove the backrest screw ("H") located on the side of the rollator, just below the handles ("A"). Slide the included backrest into the open slot, then reinsert and tighten the backrest screw ("H") to secure it. Repeat the same on both sides.

4. Attach the bag ("D"). Hook the tabs of the bag into the hooks provided.

Assembly instructions

5. To use the rollator, make sure that the front wheel lock mechanism ("E") is fully folded backward. In this position, the mechanism is released, the wheels can be turned 360 degrees, and the rollator can be used to its full extent.

Adjusting the height

Press the gray button on the height adjustment ("I") on the frame of the rollator and adjust the handle to the appropriate height. Repeat for the other side. Make sure the gray buttons on both handles are locked. Make sure that both handles are set to the same height.

Handle height: 79 - 94 cm

Your ideal handle height

Adjust the handles ("A") of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle, your elbow should be slightly bent. If you find that you need to bend over or keep your arms bent at a right angle, adjust the height of the handles.

Brakes

1. Pull the brake lever ("G") up to brake the rollator.

2. Push the brake lever ("G") down to lock the rear wheels. In this position, the parking brake of the rollator is active.
To release the parking brake again, pull the brake handles ("G") upward.

Adjusting the brakes

To adjust the rear brake ("K"), adjust the adjustment screw and locking nut. These are located on the rear wheels ("M") of the rollator. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.

! Important

If the brake adjustment has been made so that the rear brake ("K") is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

Removable cane holder

1. Attach the cane holder ("L") to the side of the frame in the provided position.

2. To do this, slide the cane holder ("L") onto the holder from above.

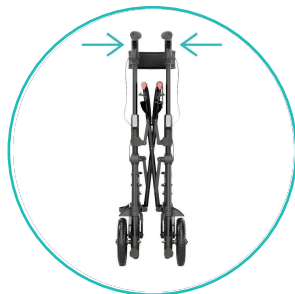
Folding and storing the rollator

1. Lock the front wheel mechanism ("E"). To do this, flip the gray mechanism completely forward. In this position, the wheels can no longer rotate 360 degrees and the rollator remains stable.

2. Pull the handle in the middle of the seat ("C") firmly upwards to fold the rollator.

3. Continue to push the rollator on both sides of the frame.

4. Lock the position by clipping the hook onto the other side.

Maintenance

1. Check the rollator at regular intervals for loose screws and defective height adjustments ("I").
2. Periodically check for proper operation of the rear wheel brakes ("K").
3. Replace damaged or worn parts.

Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws or wheels ("F"/"M").
2. Do not use the rollator if the wheels ("F"/"M") are defective.
3. Do not use the rollator if the height adjustment ("I") of the handles does not engage properly or is defective.
4. Do not use the rollator if the brake ("G"/"K") does not work properly.
5. Do not use the rollator if it does not engage properly when folded up.

Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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