Rollator Series H1370

applies to H1370-H1379

User instructions
Bedienungsanleitung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso
Bruksanvisningar
Gebruiksaanwijzing
Instrukcja obsługi

User instructions

3		User instructions
22		Bedienungsanleitung
40		Mode d'emploi
58		Instrucciones de uso
76		Istruzioni per l'uso
94		Bruksanvisningar
97		Gebruiksaanwijzing
100		Instrukcja obsługi

Rollator Series H1370

English User instructions

Intended use

This rollator is intended to be used by persons with reduced mobility as a walking aid, as a wheelchair and for transporting smaller objects. This rollator model is designed for indoor and outdoor use. It is intended for use on solid ground by persons with a maximum weight of 500 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

Before using your rollator

Read the user instructions carefully before assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

You can assemble this rollator on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor. The rollator can be used by different people. Before use, check that the height is set correctly

Important: Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: **support@helavo-rollators.com**

Scope of Delivery:

The Helavo Series H1370 comes with a frame and the following parts:

- Removable cup holder
- Removable cane holder
- Basket
- Front wheel axles with front wheels
- Rear wheel axles with rear wheels
- Legrests
- Velcro straps
- User instructions
- Product insert
- Step aid

Description of the rollator

H1370 Series Rollators:

Helavo model series H1370 consists of the following rollators:

- H1370 Hybrid-Rollator, gray
- H1371 Hybrid-Rollator, red
- H1372 Hybrid-Rollator, green

Symbols

Manufacturer

Date of manufacture


Catalog No.

Batch code

Medical Device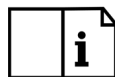

Instructions for use

CE-Marking

Maximum load capacity

Unique device identifier

Safety

Warning: The rollator can provide assistance to a person with a maximum weight of 500 lbs. The maximum load of the basket of 11 lbs should not be exceeded.

Follow the safety instructions


1. Make sure the parking brake ("L") is active when the rollator is parked
2. Check the handle height and make sure the height adjustment ("N") is locked before use.
3. Ensure that the levers are correctly attached to the mechanism and secured in place.
4. Make sure the walker is completely unfolded.
5. Keep your hands away from the folding mechanism to avoid entrapment.

What not to do


1. Do not sit on the rollator if the parking brake ("L") is not active or if it is at an angle.
2. Do not drop onto the seat ("F").
3. Do not hang any objects on the brake handles ("L") or hand grips ("K").
4. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: **support@helavo-rollators.com**

Assembly instructions: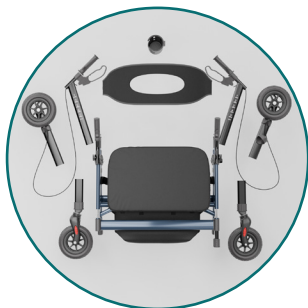


1. Remove the rollator from the packaging and remove the protective foils. Spread out the parts for assembly as shown.

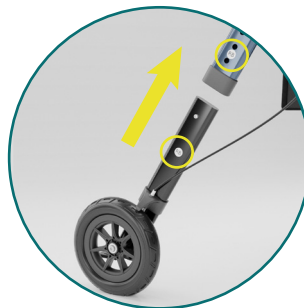2. Insert the rear wheel axle with rear wheel ("R") with the marking "RB" into the frame at the corresponding marking "RB". Both markings point to the outer side of the rollator.

3. Press and hold the two spring buttons ("O") on the sides of the rear wheel axle ("R") to slide the parts into each other. Important: The parts should click into place with a "click" sound.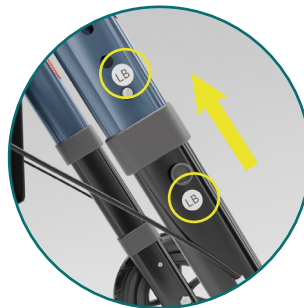

4. Repeat steps 2. and 3. for the second rear wheel axle with rear wheel ("R") marked "LB" on the other side of the frame.


Assembly instructions:

5. Insert the front wheel axle with front wheel ("T") with mark "RF" into the frame at the corresponding mark "RF". Both markings point to the outer side of the rollator.

6. Press and hold the two spring buttons ("O") on the sides of the front wheel axle ("T") to slide the parts into each other. Important: The parts should snap into place with a "click" sound.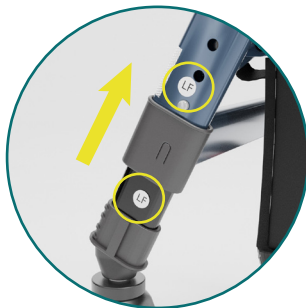


7. Repeat steps 5. and 6. For the second front wheel axle with front wheel ("T") marked "LF" on the other side of the frame.


8. Put the rollator in an upright position and unfold it. Important: Make sure the rollator is completely unfolded.


Assembly instructions: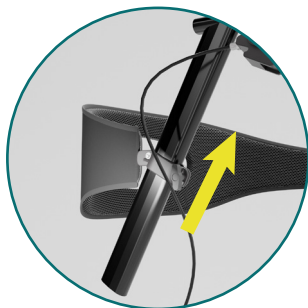

9. Attach the backrest ("A") to both sides of the handles ("K") and fix it by tightening the grey wing screws ("B"). The Helavo logo should be facing the front of the Rollator. Tip: After assembly, you can adjust the backrest to your desired height.

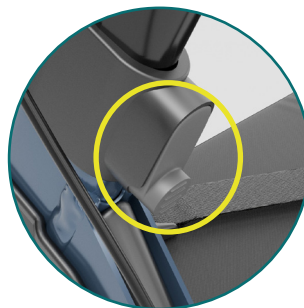10. Now insert the handles ("K") one after the other into the frame by pressing and holding the grey button of the height adjustment ("N"). Adjust the handles so that both sides have the same height.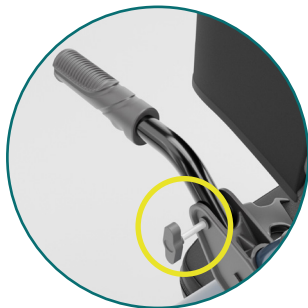

11. Add the two lower handles to frame position and fix them with the lower handle wing screw. Do this on both sides. The handles are identical on both sides and can be attached to either side. Make sure the holes align with those on the lower handle to get the wingscrew ("E") in the correct position and fastened.


12. Insert the legrest ("H") marked "RF" into the legrest attachment ("I") by pressing it down onto the mechanism.

Assembly instructions:

13. Repeat this step for the second legrest ("H") marked "LF" on the other side of the frame.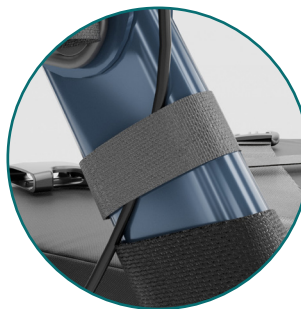

14. Our recommendation: Secure the brake cables on both sides of the frame using the velcro tape as shown.

Adjusting the height

Press the gray button on the height adjustment ("N") on the frame of the rollator and adjust the handle to the appropriate height. Repeat for the other side. Make sure the gray buttons on both handles are locked. Make sure that both handles are set to the same height.

Ensure that both handles ("K") are set to the same height

Handle height: 84.5 - 99.5 cm / 33.26 - 39.17 inch

Your ideal handle height

Adjust the handles ("K") of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle, your elbow should be slightly bent. If you find that you need to bend over or keep your arms bent at a right angle, adjust the height of the handles.


Adjusting the seat height


1. Press the spring button ("O") on the frame of the rollator and adjust the seat to the appropriate height. Repeat this for all wheel axles. Make sure all push buttons are engaged. Make sure that all wheel axles ("T" / "R") are set to the same height.

Seat height: 53 - 57.5
cm / 20.86 - 22.63 inch

Brakes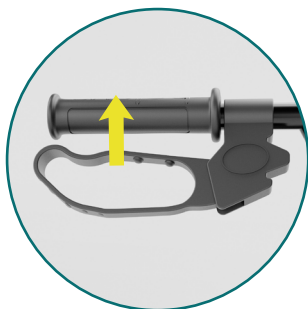


1. Pull the brake handles ("L") up to brake the rollator.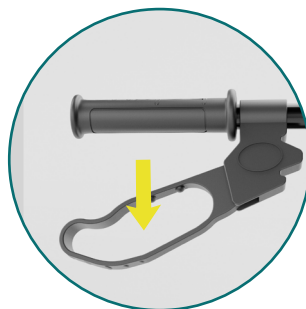

2. Push the brake handles ("L") down to lock the rear wheels. In this position, the parking brake of the rollator is active. To release the parking brake again, pull the brake handles ("L") upward.

Adjusting the brakes

To adjust the rear brake ("Q"), adjust the adjustment screw and locking nut. These are located on the rear wheels ("R") of the rollator. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.

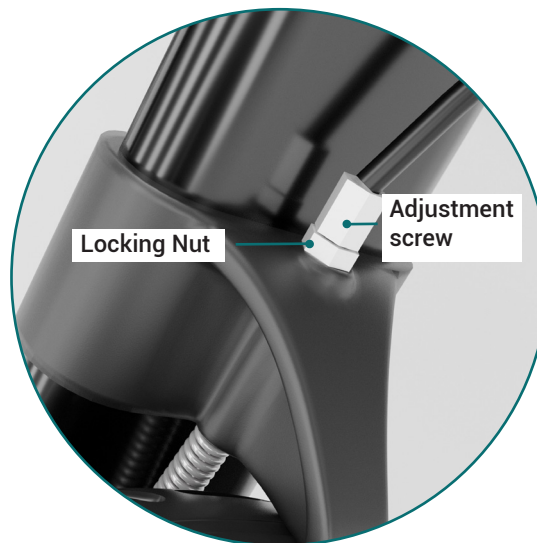Important

If the brake adjustment has been made so that the rear brake ("Q") is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

Removable cane holder


1. Attach the cane holder ("P") to the side of the frame in the provided position.


2. To do this, slide the cane holder ("P") onto the holder from above.

Removable cup holder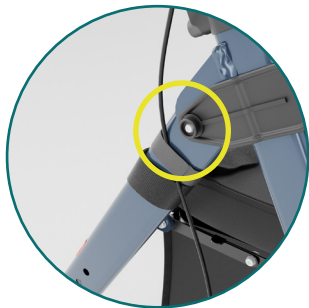


1. Attach the cup holder ("D") to the side of the handle in the provided position.

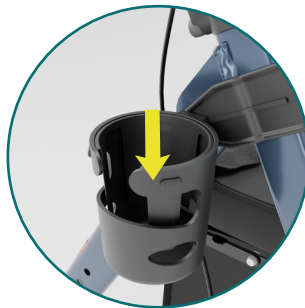2. To do this, slide the cup holder ("D") onto the holder from above.

Folding and storing the rollator

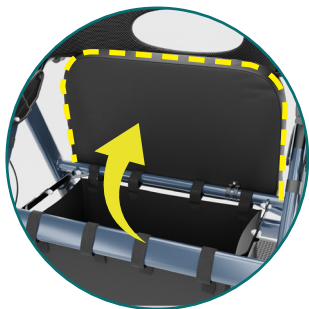1. Lift the seat ("F") of the rollator.

2. Pull up the black metal brace inside the basket to fold the rollator.

Conversion of the rollator

When using the rollator in its conventional function as a rollator, make sure that:

1. The backrest ("A") faces the front wheels.
2. The legrests ("H") are removed or attached to the rear of the rollator in the center

Converting the rollator for use as a wheelchair

1. Fold the backrest ("A") to the other side, facing the rear wheels.
2. Lift the legrests ("H") and place them on their side facing forward on the leg rest attachment part. Push them all the way down so that they are securely attached.
3. Fold down the footrests ("J") one after the other after you have sat down of the wheelchair ("F").

Important

Make sure you sit down safely on the seat ("F") and also stand up safely again. First sit down and then fold the footrests ("J") down. Fold the footrests up first and then stand up.

Rollator

Wheelchair

Maintenance

1. Check the rollator at regular intervals for loose wing and locking screws ("B"/"E") and defective height adjustments ("N").
2. Periodically check for proper operation of the rear wheel brakes ("Q").
3. Replace damaged or worn parts.

Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws or wheels ("B"/"E"/"T"/"R").
2. Do not use the rollator if the wheels ("T"/"R") are defective.
3. Do not use the rollator if the height adjustment ("N") of the handles does not engage properly or is defective.
4. Do not use the rollator if the brake ("L"/"Q") does not work properly.
5. Do not use the rollator if it no longer engages properly when unfolded.

Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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- H1370 Hybrid-Rollator, grau
- H1371 Hybrid-Rollator, rot
- H1372 Hybrid-Rollator, grün

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