Electric Wheelchair H2010

User instructions

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Mode d'emploi

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Bruksanvisningar

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User instructions

3

USER INSTRUCTIONS

41

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79

MODE D'EMPLOI

117

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155

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193

BRUKSANVISNINGAR

231

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269

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Table of contents

1. Introduction	4
2. Intended Use	4
3. Symbols Legend	5
4. Safety Warnings	6
5. Technical Parameters	12
6. Scope of delivery	13
7. Description of the electric wheelchair	14
8. Assembly & Setup	15
8.1. Folding and Unfolding the Wheelchair	16
8.2. Footrest Adjustment	17
8.3. Connecting the Battery and Motor Terminals	17
8.4. Install & Connect the Controller	18
8.5. Install the Anti-tip wheels	19
8.6. Raising and Lowering the Armrest	21
8.7. Removing and Installing the Battery	21
8.8. Driving Modes: Electric & Manual	23
8.9. Description of Controller	24
8.10. Battery and charging information	27
8.11. Using the Telescopic Towing Handle	30
8.12. Seat belt	31
8.13. Using the Wheelchair Safely: Entry, Exit, and Slope Navigation	31
8.14. Cane Holder – Installation & Use	31
9. Troubleshooting Guide	32
10. Electric Wheelchair Range	34
11. Maintenance and Self-Check	34
12. Storage & Transportation Guidelines	36
13. Warranty Terms and Conditions	37
14. Declaration of Conformity	40

1. Introduction

Thank you for choosing the **Helavo H2010** electric wheelchair. Please read this manual carefully before use, and be sure to use the device correctly and safely. It provides important information on the wheelchair's design, features, safety instructions, and care guidelines.

Store this manual in a location where it can be referred to whenever needed.

- This manual contains information that is protected by copyrights. It may not be reproduced in part or in whole without explicit written permission from Helavo.
- The contents of this manual may be changed in the future without notice.
- Use of this device in ways not consistent with the contents of this manual may result in damage to the device.

If you have any questions regarding the assembly, the electric wheelchair or if you are unsure about anything, please contact our customer service at the e-mail address: **support@helavo-mobility.com**

2. Intended use

Our powered wheelchair is for disabled or persons with reduced mobility as a walking vehicle, suitable for both indoor and outdoor environments. It is intended for use on solid ground by persons with a maximum weight of 130 kg.

The user should have the necessary physical, visual and cognitive abilities to operate the wheelchair safely.

Clinical Evaluation and Performance

In accordance with Annex XIV of Regulation (EU) 2017/745 (MDR), the Helavo H2010 electric wheelchair has demonstrated safe and effective performance in its intended use environment, as supported by user feedback, technical testing, and post-market surveillance data.

Company Headquarters:
Helavo GmbH
Address: Hauptstraße 46, 64342, Seeheim-Jugenheim,
Germany
Website: www.helavo.de

3. Symbols Legend

This manual and packaging use specific symbols to highlight important information. Each symbol is defined below—please pay close attention to their instructions.

WARNING: Indicates a potentially hazardous situation. Failure to follow the specified instructions may result in personal injury, equipment damage, or product malfunction.

PROHIBITED: The following actions are strictly forbidden under all circumstances. Performing them may result in personal injury and/or damage to the equipment.

Recyclable — Dispose of responsibly through local recycling systems.

It is prohibited to dispose with household waste

Manufacturer

CE-Marking

Date of manufacture

Maximum load capacity

Catalog No.

Keep Dry

Batch code

Fragile

Medical Device

Unique device identifier

Consult Instructions for use

4. Safety Warnings

The terms “Warning” and “Attention” in this manual highlight operations that could lead to personal injury or property damage if not followed properly.

⚠ Warning! Do not operate the wheelchair without reading the manual thoroughly. Please read the following warnings carefully to avoid damage or personal injury caused by improper operation.

⚠ Warning! Wheelchair users should take appropriate safety measures and obey local pedestrian traffic rules. The company is not responsible for personal injury and/or product damage caused by improper operation. When you sit in the wheelchair, it may be difficult for others to see you. Use of safety equipment such as reflectors, reflective clothing, lights and safety signs is critical to your safety.

⚠ Warning! Please consult your health care professional regularly to ensure that there are no health or physical conditions that limit or compromise your safe operation of the wheelchair. If you are taking prescription or over-the-counter medications, or if physical activity is limited, please consult your physician. Some medications and medical conditions may impair your ability to operate the wheelchair.

⚠ Warning! All wheelchairs are not the same. Please take time to learn before driving.

⚠ Warning! Always turn off the power before entering or exiting the wheelchair.

⚠ Warning! Being able to operate a wheelchair safely needs time and practice. Before becoming comfortable with driving, practice in a safe, open area.

Please drive smoothly and avoid weaving or making sharp turns. Ensure the pneumatic tire pressure is properly maintained, as improper pressure can lead to unstable driving and increased power consumption.

⚠ Warning! Please do not use the wheelchair for other purposes, applications such as weight training, sports and athletics, traction, moving or dragging may cause safety hazards and excessive stress in wheelchairs.

⚠ Warning! This wheelchair is for one person only, the maximum load weight is 130kg with no passengers allowed. Do not carry anything to avoid overloading and possible rollover.

⚠ Warning! Before getting on or off the wheelchair, sit as far back as possible to prevent injury from tipping over.

⚠ Warning ! The armrests are not designed to be used as transfer bars. Do not use them to transfer in or out of the chair, as this may cause damage or lead to personal injury.

Also, avoid placing your full weight on the footrest, as this may cause the chair to tip over, resulting in injury.

⚠ Warning: Be extremely cautious when using oxygen near electrical circuits or combustible materials. Contact your oxygen supplier for detailed information on the safe use of oxygen.

⚠ Warning: To prevent the wheelchair from rolling or losing control, do not place it in manual mode on an inclined surface.

⚠ Warning: If you plan to remain seated in one position for an extended period, turn off the power to prevent accidental movement of the wheelchair.

⚠ Warning: Keep yourself, your clothing, and other objects away from the wheels while the wheelchair is in motion. Do not drag anything behind the wheelchair while driving. Loose clothing or items may get caught in the tires or moving parts, leading to damage or injury.

⚠ Attention: Do not connect any other devices to the electronic system or use the wheelchair battery to power external devices.

⚠ Attention: When disconnecting the wiring harness, hold the connector firmly to avoid damage. Do not pull on the wiring directly.

⚠ Warning: Keep all charger connector terminals clean and dry. Avoid exposure to moisture to prevent damage to the electrical system or risk of personal injury.

⚠ Warning: Ensure the safety of the wheelchair and battery during transportation. Never transport the wheelchair or battery with combustible materials.

Precautions for Uphill and Downhill Driving:

To ensure safety when navigating slopes or uneven terrain, please observe the following guidelines:

- Avoid steep inclines, tilted surfaces, high steps, deep ditches, and channels. The recommended maximum slope angle is less than 9°.
- When driving on a slope, always move straight up or down—never at an angle—to prevent tipping.
- Do not drive sideways across slopes or tilt the wheelchair by climbing with one side first.
- Slow down when descending. If the wheelchair picks up speed unexpectedly, release the joystick to stop, then gently push it forward to regain controlled movement.
- Never switch to manual mode while on a slope.
- Do not attempt to climb stairs or cross high steps.
- When crossing ditches, ensure the wheels approach at a 90-degree angle for stability.

Note: When driving on a slope, releasing the joystick may cause the wheelchair to briefly roll backward. Whether moving forward or in reverse, the wheelchair may roll back approximately 30 cm before the brakes fully engage.

 **Warning:** Always keep both front wheels in contact with the surface when climbing. Approaching a slope at an angle or unevenly may increase the risk of tipping.

 **Warning:** Do not attempt to climb over obstacles or cross ditches higher than 40 mm. Doing so may result in instability or injury.

 **Warning:** Reverse driving is only permitted on flat surfaces. Operate slowly and smoothly while reversing, and stop frequently to check for obstacles. To prevent rollover, do not reverse downhill or on a slope.

Warning: Hazardous Driving Conditions:

To avoid accidents, injury, or damage to the wheelchair, do not operate the wheelchair under the following conditions:

- On hazardous surfaces such as snow, ice, wet grass, fallen leaves, mud, sand, gravel, or uneven terrain, which may cause loss of traction or control.

- In bad weather, including rain, heavy fog, strong winds, or snow. This electric wheelchair is not waterproof. Exposure to moisture can damage the electronics and increase the risk of short-circuiting or malfunction. If the wheelchair becomes wet, dry it immediately to prevent damage.
- Near bodies of water such as streams, lakes, ponds, or the sea, and never attempt to drive through water.
- In areas with heavy pedestrian or vehicle traffic, where maneuvering could be dangerous.
- Along open edges without protective barriers, such as ditches or elevated platforms.
- Near or across railroad tracks. If crossing is absolutely necessary, proceed with extreme caution:
 1. Stop and assess the situation first.
 2. Cross tracks at a right angle.
 3. Ensure the wheels do not get caught between or on the rails.

⊘ PROHIBITED! Do not expose the wheelchair to open flames or high heat sources, as this may cause fire or damage.

⊘ PROHIBITED! Keep your wheelchair in a dry and clean environment. Do not use the wheelchair in the shower, bath, pool, or sauna.

⊘ PROHIBITED! Do not attempt to use the wheelchair on stairs or escalators.

⚠ Warning: For safe handling, always turn off the power before moving the wheelchair. Accidental contact with the joystick may cause unexpected movement. Ensure the brakes are locked to prevent the wheelchair from rolling during handling.

⚠ Warning: When entering an elevator, the wheelchair may be driven inside, but the power must be turned off immediately, and the user should sit firmly to ensure the wheelchair remains stationary.

⚠ Warning: Do not allow children to play with or operate the wheelchair.

⚠ Warning: This wheelchair must not be used inside an MRI room when in a hospital setting.

⚠ Warning: If the wheelchair requires repair or modification, please contact the customer service center. Do not attempt to modify it yourself, as this may lead to malfunctions or accidents.

⚠ Warning: Electric wheelchairs must not be transported with the brakes disengaged. We recommend placing the wheelchair in the vehicle's trunk and securing it firmly to prevent movement during transport.

⚠ Warning: Do not lean forward or bend down to pick up objects while seated in the wheelchair, as this may cause the wheelchair to tip and result in injury.

⚠ Warning: After any significant impact, fall, or crash, inspect the wheelchair thoroughly for structural or electronic damage before continued use. Contact a technician if unsure.

⚠ WARNING: Electromagnetic Interference (EMI):

Sources of radio waves—such as radio and TV stations, amateur radio transmitters, two-way radios, and mobile phones—can interfere with the control system of your power wheelchair.

To reduce the risk of unexpected movement or braking, which may result in serious injury, please follow these precautions:

1. Do not use portable communication devices (e.g., CB radios, mobile phones) while the wheelchair is powered on.
2. Avoid close proximity to strong transmitters, such as broadcasting towers or other high-power radio sources.
3. If the wheelchair behaves unexpectedly or releases its brake without warning, immediately switch off the power if it is safe to do so.
4. Do not modify or add unauthorized accessories to the wheelchair, as this may reduce its resistance to electromagnetic interference.
5. Report any incidents of unexpected movement or brake failure to the manufacturer. Always remain alert to nearby radio sources and follow all safety advisories.

This symbol indicates that, in accordance with local laws and regulations, this product must not be disposed of with household waste. When the wheelchair reaches the end of its useful life, please take it to a designated collection point for the recycling of electrical and electronic equipment. Proper disposal helps conserve natural resources and ensures that the product is recycled in a manner that protects human health and the environment.

Important Disposal Guidelines:

- **Electrical Components:** The wheelchair contains electrical and electronic parts that fall under the scope of the WEEE Directive. These components must be disposed of through authorized recycling facilities equipped to handle electronic waste.
- **Batteries:** If the wheelchair includes lithium-ion batteries, they must be removed and recycled separately in accordance with local regulations governing battery disposal.
- **Plastic and Metal Parts:** Non-electronic components, such as plastic covers, the vehicle frame, and pipe structures, should be taken to professional recycling centers that can process these materials appropriately.

Contraindications and Residual Risks

This wheelchair is not recommended for users who:

- Have recently undergone surgery affecting balance or movement
- Suffer from severe cognitive impairments or disorientation
- Have significant uncorrected visual impairments
- Require the use of an intravenous pole, oxygen tanks (unless used per guidelines), or other attachments without proper integration
- Weigh more than 130 kg
- Experience frequent loss of consciousness or seizures

Residual Risks remain even with correct usage, including:

Residual risk that cannot be fully eliminated	Practical mitigation / user action
Tipping on steep or uneven terrain	• Keep to gradients $\leq 9^\circ$ and drive straight up/down slopes, never sideways or diagonally • Always deploy the supplied anti-tip wheels; never remove them • Reduce speed when descending; avoid high kerbs, stairs and loose surfaces
Reduced responsiveness due to electromagnetic interference (EMI)	• Do not use CB radios, mobile phones, or other transmitters while the chair is powered on • Stay clear of strong broadcast towers; switch off the chair immediately if it behaves unpredictably
Battery overheating if stored or used improperly	• Keep batteries away from direct heat sources and open flames; store/charge at 5 – 40 °C • Follow the charging schedule and disconnect the charger once the green LED is lit
Skin injury from prolonged seating without repositioning	• Shift weight (or perform a brief lift) at least every 30 min • Follow clinical advice on sitting time limits and schedule regular pressure-area checks with a healthcare professional

5. Technical Parameters

Model	H2010
Body Frame	Aluminum
Maximum Load Capacity	130 kg
Input power (VA)	≤500
Length L (mm)	1000±30
Width B (mm)	660±30
Height H (mm)	970±30
Folded size	660 x 360 x 840 mm
Net weight (without battery)	24 kg
Battery Weight	2 kg/ piece
Maximum Speed forward	Up to 6 km/h
Maximum Speed backward	Less than 3 km/h (0.8 m/s)
Horizontal road braking	≤1.5m
Max safe slop braking	≤3.6m (6°)
Slope performance	9°
Dynamic stability	≥6°
Span obstacle height	≥40mm
Foot height off ground	100mm
Min Turning Radius	≤950mm
Front wheel size/type	8" (20cm) PU Solid tire
Rear wheel size/type	12" (30 cm) Pneumatic tire
Battery capacity	Li-ion Battery pack (25.2V 10.4Ah x 2), UN38.3 certified
Brake system	Automatic electromagnetic brake system
Motor	Brushless DC motor; 24V DC;150W, 2pcs
Output voltage	24V DC
Safety type	II B
Degree of protection	IP24

6. Scope of delivery

The Helavo H2010 comes with the following parts:

1. Electric wheelchair

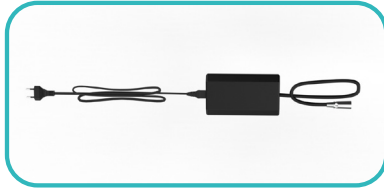
7. Cane holder

2. Lithium batteries 25.2V 10.4Ah x 2 pcs

8. Side Bag

3. Charger

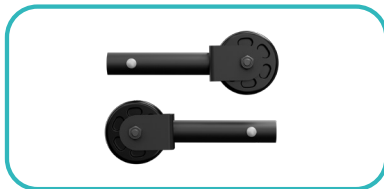
9. Under seat storage bag

4. Controller and Knob Screw

10. Tool Kit

5. Anti-tip wheels

11. User instructions

6. Seat and backrest cushions

12. Product inserts

7. Description of the electric wheelchair

Normal Operating Conditions:

The electric wheelchair is designed to function properly under the following environmental conditions:

- Ambient Temperature: 5°C to 40°C
- Relative Humidity: $\leq 80\%$
- Atmospheric Pressure: 80 kPa to 105 kPa
- Input Voltage (Charger): AC 220V $\pm 22\text{V}$, 50Hz $\pm 1\text{Hz}$
- Output Voltage (Charger): DC 24V $\pm 2.4\text{V}$

8. Assembly & Setup

Installation Hint:

- No tools or additional equipment are required to assemble the electric wheelchair.
- Before opening the box, inspect the carton for any signs of damage or crushing.
- Read all provided documentation thoroughly to become familiar with the product's assembly and operation procedures.
- Unpack the contents carefully, removing the main body of the wheelchair, seat cushion, user manual, and any other included materials.
- Gently remove all protective materials, such as film, plastic, and foam, used to safeguard the components during shipping.
- Carefully cut the zip ties with scissors to release the wheelchair from its shipping position.

Fig.2

Please keep this **packaging and protective inserts** for the entire warranty period; they're designed to safeguard the wheelchair in the event of service or warranty return.

8.1. Folding and Unfolding the Wheelchair

A.Unfolding:

When you first receive the wheelchair, it will be folded inside the box. To unfold it, place the wheelchair flat on the ground with the wheels facing downward.

Hold the top of the backrest with one hand and the front edge of the seat with the other. Gently pull them apart until the wheelchair is fully unfolded—you should hear a click when it locks into place.

Fig.3

⚠ Make sure the latch on the back of the wheelchair is locked.

B.Folding:

To fold the electric wheelchair, pull the handle located beneath the backrest outward (see Figure 4), then push the backrest down toward the seat in the direction indicated by the arrow in Figure 5.

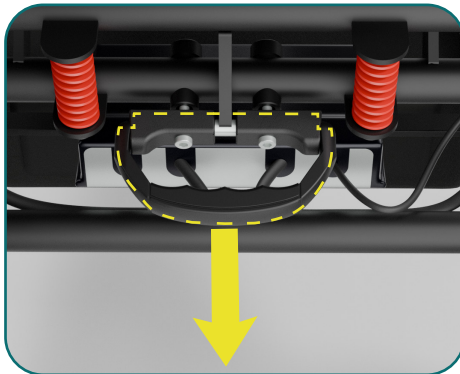
Fig.4

Fig.5

8.2. Footrest Adjustment

Lower the footrest from its vertical position for proper support during use.

Fig.6

8.3. Connecting the Battery and Motor Terminals

Under the seat, you will find two red connectors. These are intentionally disconnected during transport for safety reasons.

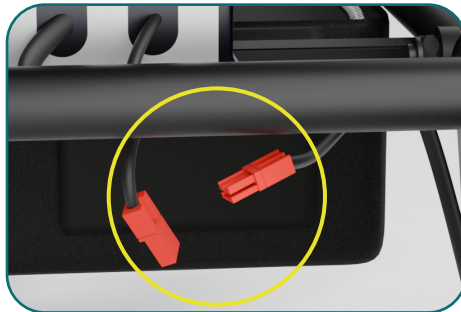
Fig.7

Before using the electric wheelchair, ensure both red connectors are securely plugged in, as shown in the figure. This connects the battery to the motor system and enables normal operation.

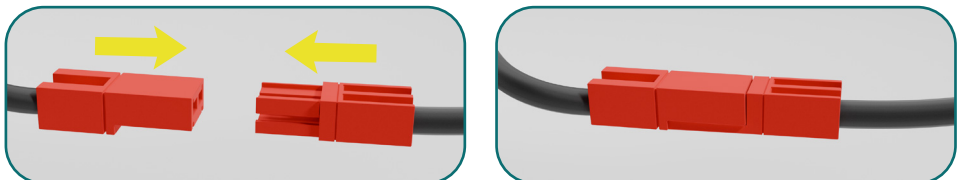
Fig.8

8.4. Install & Connect the Controller

A.Installing the Controller to the Wheelchair.

Loosen the knob located beneath the right armrest, insert the controller tube into the armrest mounting tube, adjust it to the desired position, and then securely tighten the knob.

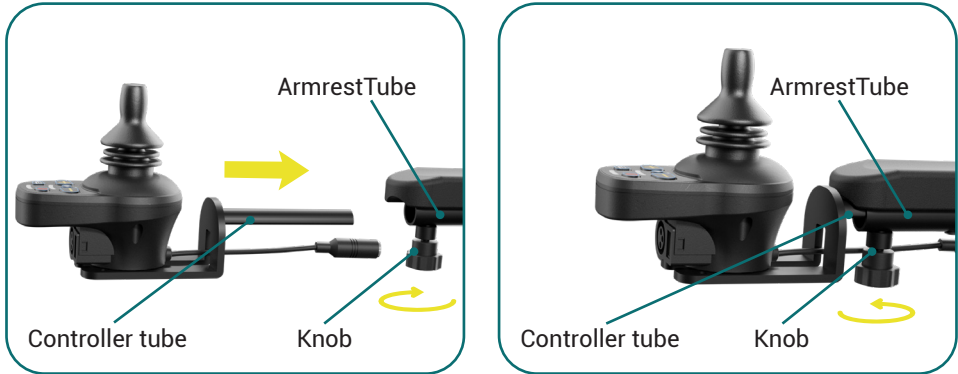
Fig.9

Note: The controller can be mounted on either the left or right armrest, depending on the user's preference.

B.Connecting the Controller to the Wheelchair.

Locate the controller cable and the corresponding socket on the wheelchair's control module.

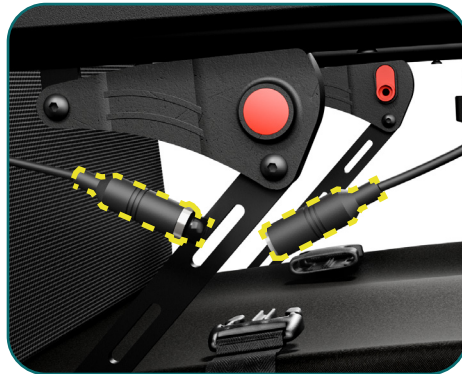
Fig.10

Align the Connectors: ensure the connector pins are properly aligned to avoid damage—make sure the **indicator arrows on both connectors are perfectly aligned** before connecting. Gently insert the controller cable into the socket until it is fully seated.

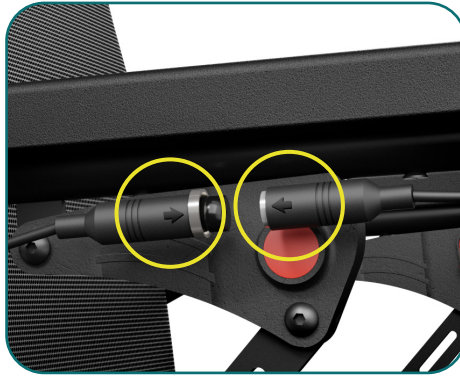
Fig11

Secure the Connection: rotate the screw collar, to secure the connection and prevent accidental disconnection during use.

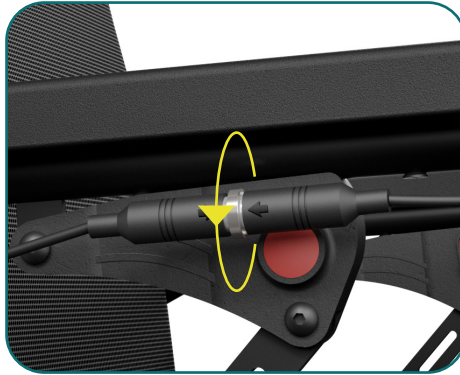
Fig12

8.5. Install the Anti-tip wheels

Identify the mounting tubes at the lower rear of the wheelchair frame.

Fig13

While pressing the locking pin, insert the anti-tip wheel tube into the frame tube, aligning it with the locking hole.

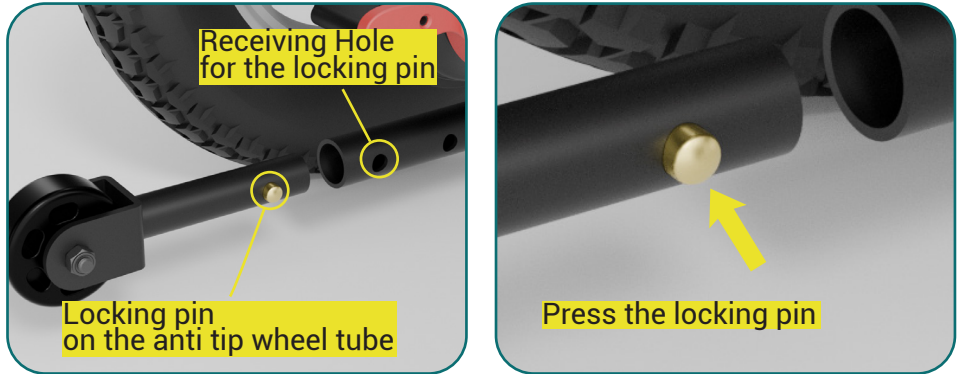
Fig14

Release the locking pin once the tube is fully inserted. It should click into place to secure the wheel.

Fig15

Repeat the process for the second anti-tip wheel.

Once both anti-tip wheels are installed, gently pull on them to ensure they are locked firmly and do not slide out. Confirm they are level with the ground and aligned properly.

⚠ Warning: Always use both anti-tip wheels. Operating without them may lead to tipping and serious injury.

8.6. Raising and Lowering the Armrest

To lift the armrest, press the release button while holding the armrest tube, then raise it upward (See Figures 16 and 17). This allows easier access when getting on or off the wheelchair.

Before driving, always lower the armrest back into place and ensure it is securely locked.

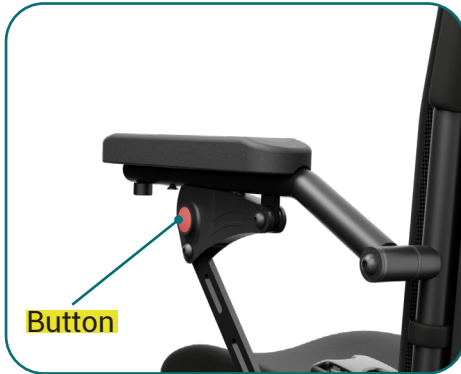
Fig16

Fig17

8.7. Removing and Installing the Battery

⚠ Attention! Always ensure the controller is turned off before removing or installing the batteries to prevent any unintentional movement of the wheelchair.

To remove the battery:

Press the red side button, then lift the battery handle upward at a slight angle to detach the battery from the frame.

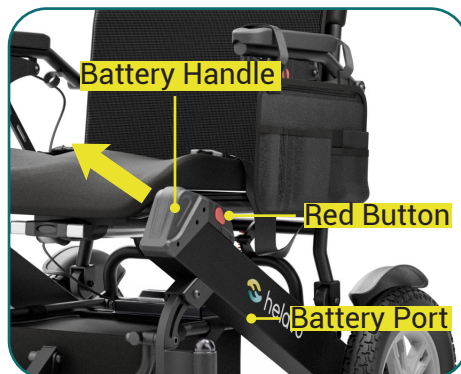
Fig18

To install the battery:

Align the battery's connector with the insertion port on the frame. Raise the front corner of the seat cushion to clear the path and slide the battery downward firmly until it is fully seated.

Fig19

⚠ Attention! Ensure the red button pops back up, confirming the battery is securely locked in place.

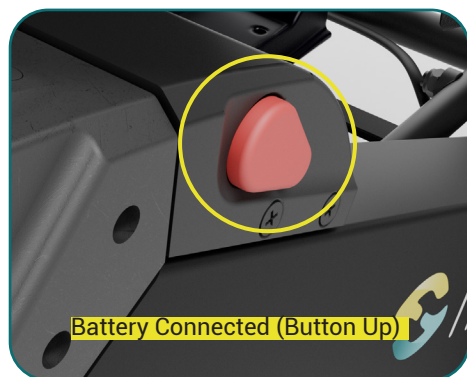
Fig 20

⚠ Important Note: This electric wheelchair is equipped with two batteries and two corresponding slots; however, **only the right-side slot is active** for powering the wheelchair. When using the chair outdoors and recharging is not immediately possible, you will need to **manually move the charged battery from the left slot to the right slot** once the main (right-side) battery is depleted. Be sure to recharge both batteries as soon as possible after use.

8.8. Driving Modes: Electric & Manual

This electric wheelchair features two driving modes:

- **Electric Mode** – The wheelchair is powered and controlled using the joystick. In this mode, it cannot be pushed manually.
- **Manual Mode** – The wheelchair can be pushed by an assistant. The joystick is disabled and does not control movement.

⚠ Warning: Never switch to Manual Mode on an incline or slope, as the wheelchair may roll uncontrolled, leading to injury.

To switch between modes, use the red levers located at the lower rear of the wheelchair:

- Levers **down**--Electric Mode
- Levers **up**--Manual Mode

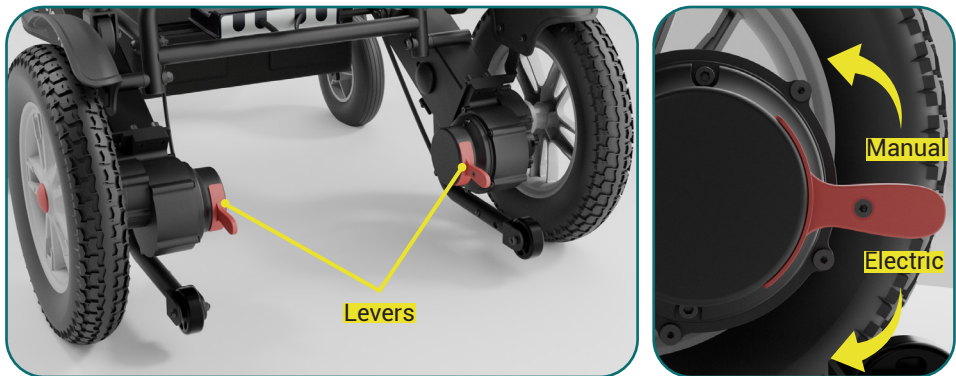
Fig. 21

⚠ Attention: Switching Between Modes

Before switching between Manual and Electric Mode (by moving the levers up or down), always make sure to turn off the controller first.

If you accidentally switch modes while the controller is still on, the wheelchair may not respond when switched back to Electric Mode. In this case, follow these steps:

1. **Turn off the controller** immediately.
2. Set the levers to **Electric Mode** (lever position down).
3. **Wait a few seconds.**
4. Turn the controller back on.

The wheelchair should now function normally in Electric Mode.

Note: Electric Mode Also Acts as a Brake

When the wheelchair is in Electric Mode (levers down), the motors remain engaged and act as an automatic brake, preventing the wheelchair from rolling.

If you're using the wheelchair in Manual Mode (levers up) and need to park it or leave it stationary for an extended period, it is recommended to switch back to Electric Mode to activate the braking function and keep the wheelchair securely in place.

8.9. Description of Controller

The joystick unit integrates all essential electronic controls needed to operate the wheelchair. It features the following components:

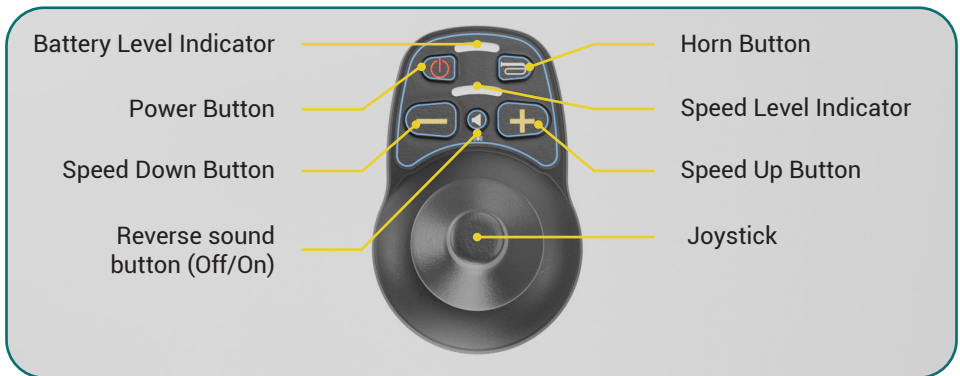
Fig. 22

⚠ Attention: Before turning on the controller, ensure that the joystick is in the center (neutral) position. Starting the chair with an off-center joystick can cause sudden, unintended movement.

- **To turn on the controller**

Press the Power button. The battery and speed indicator lights will turn on, and a beep from the buzzer will confirm that the controller has powered on successfully.

- **To turn off the controller**

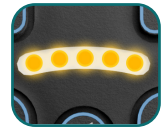
Press the Power button again. All indicator lights will turn off, and the controller will cease power output.

⚠ In case of an emergency, you can press the Power button at any time to immediately shut off the system.

- **Adjusting the Speed:**

You can adjust the maximum driving speed of the electric wheelchair according to your personal preferences and environment. After powering on, the **Speed Indicator** displays the current maximum speed level using a series of bars.

-Press the **Speed Down Button** to decrease the speed by one level.

-Press the **Speed Up Button** to increase the speed by one level.

Each press adjusts the number of illuminated bars, which represent the current speed setting. There are five speed levels available:

- Level 1: 2 km/h ●
- Level 2: 3 km/h ● ●
- Level 3: 4 km/h ● ● ●
- Level 4: 5 km/h ● ● ● ●
- Level 5: 6 km/h ● ● ● ● ●

Note: Speed tolerance is within ± 0.5 km/h. The user can adjust the speed range at any time.

- **Horn Activation:**

To use the horn, press and hold the horn button . The horn will sound continuously while the button is pressed and will stop once released.

- **Activating or Deactivating the Rear Sound Alert**

If desired, you can mute the Rear Sound Alert by pressing this button once. To turn the sound back on, simply press the same button again.

Note: By pressing this mute button, the other Controller sounds—such as those produced when changing the speed or turning the wheelchair on—will also be muted.

- **Battery Level Indicator**

The battery level indicator illuminates when the wheelchair is powered on, showing the estimated remaining battery charge:

-**All LEDs on (Green):** Battery is fully charged.

-**Two or three LEDs on:** Battery level is low. Recharge soon—especially before longer trips.

-**One LED on (Red):** Battery is critically low. Recharge immediately to avoid interruption.

For best performance and to avoid unexpected shutdowns, always recharge the battery before extended use.

• Joystick Operation

The joystick controls both the **direction** and **speed** of the electric wheelchair. The further you push the joystick in any direction, the faster the wheelchair will move in that direction.

- Push **forward**--move forward
- Pull **backward**--move in reverse
- Push **left**--turn left on the spot
- Push **right**--turn right on the spot

The joystick provides **smooth 360°** control, allowing the wheelchair to move in any direction intuitively—based on both the angle and pressure applied.

When the joystick is released, it automatically returns to the center position and activates the **braking system**, bringing the wheelchair to a safe stop.

Controller Beep Code Diagnostic Table

When a fault occurs, the controller emits a series of beeps. Use the number of beeps to identify the issue and follow the suggested solution below:

Beep Count	Fault Description	Recommended Action
1 beep	Low Battery Voltage	Battery voltage is too low. Try charging the battery. If the issue persists, replace the battery.
2 or 3 beeps	Left Motor Error	Check the left motor, its wiring, and connectors for loose or damaged connections.
4 or 5 beeps	Right Motor Error	Inspect the right motor and all related wiring and connectors for damage or looseness.
6 beeps	Overcurrent Detected	Check if the motor is jammed or obstructed. If not, the controller may be faulty.

7 beeps	Joystick Error	Joystick may be off-center, damaged, or have a loose or broken cable. Inspect thoroughly. If the joystick remains off-center when released, power off the controller, ensure it's not physically obstructed, and restart the wheelchair. If the issue persists, contact service.
8 beeps	Controller Error	Ensure the battery is fully charged and check all connections. If unresolved, contact a service technician.
9 beeps	Communication Failure	Internal communication fault. Contact an authorized service provider.

8.10. Battery and charging information:

The charger has a built-in indicator light with two status modes:

- 1.Red – Charging in progress
- 2.Green – Charging complete (when the charger is connected to the power outlet plus to the wheelchair/ battery as well).

Charging Methods:

Method 1 - Charging through the controller

⚠ Attention! Before charging, ensure that the wheelchair controller is turned off.

Plug the metal 3-pin connector of the charger into the 3-hole charging port on the controller. Then, insert the other end of the charger into a wall power outlet.

Fig 23

Attention! When charging through the controller port, only the right-side battery (main battery) will be charged.

To charge the **left-side battery**, follow these steps:

- **Unplug the charger** from the power outlet.
- **Swap the positions** of the two batteries (place the left battery on the right side).
- **Reconnect the charger** to the power outlet.
- Continue charging until the **green light** on the charger indicates a full charge.

Method 2 - Charging the battery separately

Remove the battery from the wheelchair and open the circular charging port cover at the bottom.

Insert the charger plug into the battery port and connect the other end to a power source.

- The **red light** indicates charging mode.
- The **green light** indicates full charge.

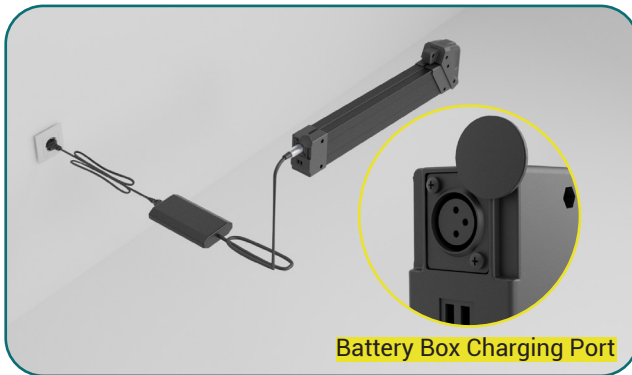
Fig24

⚠ Charging Guidelines and Safety Notes:

⚠ Warning: Always charge the battery in a well-ventilated area, away from flammable materials, heat sources, or direct sunlight. Charging in enclosed or poorly ventilated spaces may lead to heat buildup and increase the risk of fire or explosion. Never charge the wheelchair near curtains, bedding, or other flammable items.

⚠ Warning: Keep children and pets away from the wheelchair during charging. Unsupervised contact during charging poses electrical, fire, and injury risks.

• To ensure safety and maintain your product warranty, only use the original charger provided with your electric wheelchair.

Using unauthorized chargers may: Damage the battery or wheelchair, Void your warranty, Increase the risk of fire or serious malfunction.

- Do not leave the charger connected to both the power source and the batteries for more than 24 hours. Typically, one battery requires approximately 3 to 4 hours to reach a full charge. Extended connection beyond 24 hours is unnecessary and may affect battery performance over time.

- Avoid disconnecting the battery before it is fully charged (indicated by the green LED on the charger), as doing so may reduce battery lifespan.

- After charging, always disconnect the charger from both the power outlet and the controller to avoid unintended power drain.

- If the wheelchair is not in use for extended periods, recharge the batteries at least once a month to preserve battery health and prevent capacity loss.

- The electric wheelchair is equipped with a safety circuit to prevent movement during charging.

If the wheelchair does not operate properly after charging, check that the charger has been fully removed from the charging port.

- The charger must be stored in a dry, protected area with a temperature between -25°C and 40°C, and kept free from mechanical damage.

- Repairs or servicing of the charger must only be performed by an authorized dealer.

- Charge the battery regularly, even when the wheelchair is not in use, to prevent deep discharge and extend battery life.

- If storing the wheelchair for over 5 days, charge the battery for 12 hours, unplug the charger, and ensure the battery is disconnected.

- Avoid overcharging or charging under full load, as this can reduce range and cause permanent battery damage.

- Inspect batteries regularly for visible damage. Contact your dealer immediately if any issues are found.

Important: Avoid short-circuiting battery terminals. Remove all metal jewellery (e.g., watches, necklaces) before inspecting the battery.

Battery Handling and Safety Warnings

- Handle and dispose of batteries and chargers with care.

- Do not incinerate batteries. Dispose of them according to local environmental regulations.

- Keep batteries away from heat sources. Overheating may lead to an explosion.

- Do not puncture, crush, or expose the battery to high external pressure. These actions may cause internal short-circuits or overheating.

- Avoid charging in environments with open flames or while smoking.

- Do not touch the metal battery terminals.

- Use only approved batteries. If the battery casing is damaged or leaking, do not attempt to repair it—contact your local dealer immediately.

- If handling a leaking or damaged battery:

- Wear insulated gloves and safety goggles

- Replace the battery immediately and dispose of it following proper guidelines

⚠ In Case of Battery Damage or Leaks

Battery fluid is corrosive:

- Avoid contact with skin or clothing – rinse immediately with soap and water.

- If it contacts the eyes, flush with clean water for at least 10 minutes and seek medical attention.

- Contact our customer service for safe removal and replacement.

8.11. Using the Telescopic Towing Handle

When the wheelchair is folded, press and hold the release button on the telescopic handle, then pull the handle upward until it fully extends (see Figure 25).

Once extended, tilt the wheelchair slightly onto its standing wheels, then use the handle to tow the wheelchair as shown in Figure 26.

Fig 25

Fig 26

8.12. Seat belt

For your safety, always wear the seat belt while operating the wheelchair. The seat belt helps prevent accidental falls or injury, especially when driving over uneven terrain or during sudden stops. Ensure it is securely fastened before each use.

8.13. Using the Wheelchair Safely: Entry, Exit, and Slope Navigation

Getting Into the Wheelchair:

- Make sure the power is turned off before getting seated.
- Carefully transfer your body into the seat. You may use a crutch or support to assist with the movement.
- Place your feet securely on the footrests.
- Press the power button to turn the wheelchair on. Use the joystick to control forward, backward, left, and right movement.

Getting Out of the Wheelchair:

- Ensure the wheelchair has completely stopped.
- Press the power button to turn off the device.
- With the help of a crutch or support, slowly transfer yourself out of the seat.

⚠ Safety Precautions: Slopes and Uneven Surfaces:

Always approach slopes head-on (never sideways) and drive slowly. Avoid steep or uneven surfaces, stairs, high curbs, or deep ditches. Never switch to manual mode while on a slope.

8.14. Cane Holder – Installation & Use

A cane holder is included as an optional accessory. Please refer to the instructions below for installation and use: The cane holder consists of 2 parts:

Bottom holder (mounts low on the backrest bar) and **Top holder** (mounts at mid-height on the backrest bar) (Fig. 27)

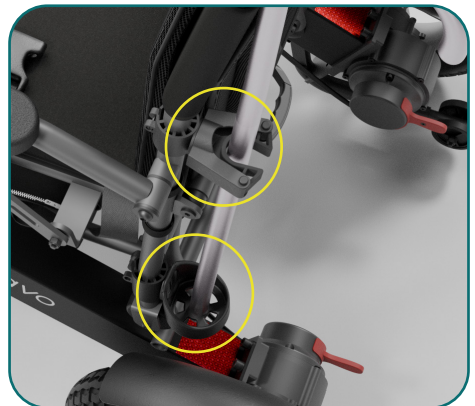
Fig. 27

Place one plastic ring around the backrest tube. Pull apart the ring ends to create enough space for the holder opening to fit between the tabs. Once in position, release the ring so it secures the holder (Fig. 28a). Then insert and tighten the screw using the provided cross-head tool (Fig. 28b). Repeat the same steps for the second part of the holder.

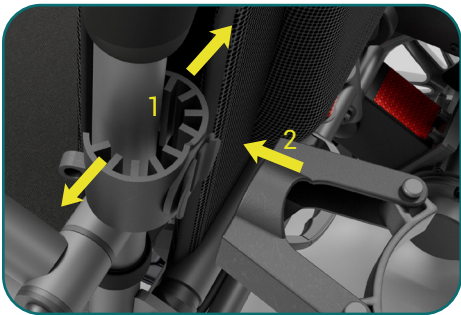
Fig. 28a

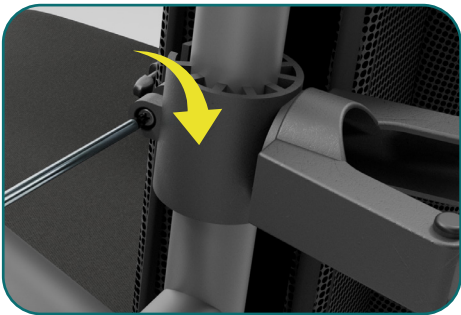
Fig. 28b

Use: Ensure the elastic strap on the top holder is positioned outward. Insert the cane tip into the bottom holder. Push the cane into the top holder until secured (Fig.29).

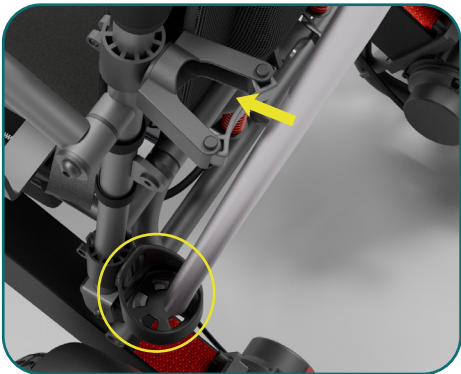
Fig. 29

9. Troubleshooting Guide

Problem	Possible Cause	Solution
Short Driving Range	Low battery charge	Charge the battery for at least 8 hours
	Battery degradation	Replace the battery
Battery Won't Charge or Shows No Power on the Meter	Battery failure	Replace the battery
	Charger malfunction	Replace the charger
	Damaged wiring or plug	Contact customer service for repair
	Loose connections	Check and reconnect cables and plugs

	No power from outlet	Try a different power socket
	Overload protector button is triggered	Turn off power and press the reset button on the battery box
Unusual Power Input (High Current)	Battery malfunction	Replace the battery
	Power was on during charging	Always turn off the wheelchair during charging
Wheelchair Not Moving	Motor brake is engaged	Release (disengage) the motor brake
	Low battery	Charge the battery
	Power is off	Turn on the power switch
	Battery not fully connected	Reinsert battery box into the frame properly
	Charger is still connected	Disconnect the charger before operating
	Overload protector triggered	Turn off power and press reset button on battery box
	Loose wiring	Check all plugs and wiring for secure connection
	Controller malfunction	Contact the service center for assistance
Motor Noise or Irregular Operation	Motor fault	Contact the service center
	Controller fault	Contact the service center

Important Note:

Do not attempt to open or modify any part of the wheelchair's electrical system, including the battery box, wiring harness, connectors, or charger. These components are not user-serviceable and should only be inspected or repaired by authorized service personnel.

10. Electric Wheelchair Range

The provided 2 x Lithium 10.4Ah 25.2V batteries have **a range of 25-30km.**

Please be aware that the driving range of an electric wheelchair can vary depending on multiple factors.

Although we follow standardized testing procedures to estimate range, real-world results can vary due to differences in motor power, total weight, and environmental conditions.

Factors That May Reduce Driving Range:

- User weight exceeds 130 kg
- Battery is worn out, improperly stored, or past its optimal use date
- Challenging or uneven terrain (e.g. slopes, mud, gravel, grass, snow, or ice)
- Frequent use on inclines or long uphill routes
- Extreme temperatures (hot or cold)
- Worn tires or high rolling resistance (e.g. thick carpets)
- Frequent starting, stopping, or sharp turns

Note: If you're driving outdoors and notice that the battery indicator light has turned red, you can help extend your remaining range by lowering the maximum speed setting.

11. Maintenance and Self-Check

Regular maintenance is essential to ensure the long-term performance and safety of your electric wheelchair. Basic tasks such as cleaning, checking tires, and battery care can be performed by the user. More complex servicing should be carried out by qualified service personnel.

Cleaning and Maintenance

•Wheelchair Frame:

-Clean the wheelchair regularly using a soft cloth to prevent dust and dirt buildup.

•Tire Inspection and Inflation

-Check the tire pressure and condition regularly.

-Replace the tire if the tread depth is less than 1 mm or if the tire shows cracks or damage.

•Inflating the rear tires:

1. Remove the valve cap by turning it counterclockwise.
2. Attach a manual air pump compatible with Schrader valves (standard car-type valve).
3. Inflate the tire to the recommended pressure.
4. Remove the pump and screw the valve cap back on securely.

⚠ **Recommended tire pressure:**

240 kPa (35 psi)

Always use a pressure gauge to ensure correct inflation.

•Battery:

- Charge the battery frequently to maintain optimal performance.
- To prolong battery life, avoid waiting until the battery is fully drained before recharging.

Routine Maintenance and Inspection

To ensure safe and reliable operation, follow this maintenance schedule:

Zone A – Daily Safety Checks	Perform these checks every time before using the wheelchair:	Backrest: Confirm it is securely locked in place Armrests: Check that they are fixed and not loose Controller: Ensure it is properly positioned and connected Footrest: Make sure it is positioned correctly Manual/Electric Mode Levers: Confirm they are in the correct position for use Battery Level: Check the power indicator on the controller
Zone B – Monthly Inspection	Perform these checks monthly to detect wear or loose components:	All screws and bolts: Check for looseness Brakes: Test for secure engagement and release Manual/Electric Mode Clutch: Ensure proper function Tire condition: Inspect tread wear; replace if worn Cable connections: Inspect the plugs for the controller and charger
Zone C – Semi-annual Professional Maintenance	Every 6 months , a complete inspection should be carried out by a qualified technician to ensure long-term safety and performance.	

12. Storage & Transportation Guidelines

Storage: If you plan to store your powered wheelchair for more than one week, fully charge the battery for up to 12 hours and then disconnect it to help prevent battery discharge during storage.

Transport: The wheelchair is designed for quick disassembly and easy transport. Follow these steps to ensure safe handling:

- Turn off the power and ensure the brake levers are engaged (locked position).
- Disconnect the Controller from the main frame.
- Remove the seat and fold the wheelchair.
- Lift the wheelchair frame into the vehicle trunk.
- Store the seat separately in a secure and protected position.

⚠ Attention: Before transport, ensure that the seat back, armrests, all foldable components are fully locked into position.

Failure to do so may result in movement during transit and cause injury or damage.

To reassemble the wheelchair after transport, simply reverse the above steps.

Additional Transport Notice:

While basic transport can be done using minimal securing methods, for maximum protection during shipping or long-distance travel, we recommend using the original packaging box and securing the wheelchair inside the transport vehicle.

Note: The battery complies with international safety standards and is generally approved for transport by air, train, and ship. Each 10.4 Ah, 25.2 V pack is 262 Wh; tested per UN 38.3; complies with IATA PI 967 Section II.

Please check with the transport provider in advance for specific requirements for lithium battery transport.

Storage & Transport Conditions

- Keep the wheelchair away from moisture, direct sunlight, and heat sources.
- Do not store outdoors or in damp environments to avoid damage to electrical components.
- Protect the unit from rain or humidity during both storage and transportation.

Environment temperature -40°C~+55°C

Relative humidity≤80%

Air Pressure 86kPa~106kPa

Storage period:5 year

13. Warranty Terms and Conditions

At Helavo, we are committed to delivering top-quality mobility solutions and exceptional customer service. Our warranty policy is designed to ensure transparency and provide you with the support you need. Please review the following terms carefully:

13.1. Warranty Coverage:

We offer a 3-year limited warranty for the wheelchair frame and 1-year limited warranty for the electric components (motors, controller, electromagnetic brakes, charger) starting from the date of delivery for the first purchaser. This warranty applies only to the originally purchased product and does not cover any replacement units provided under a warranty claim. It covers essential structural components of the electric wheelchair, covering material and workmanship defects that existed at the time of delivery. It runs concurrently with statutory warranty rights and is intended to give you additional peace of mind regarding the durability and reliability of your product.

Essential structural components covered by this warranty include:

- The frame – The central supporting structure that provides stability and durability.
- Joints and hinges – Mechanical parts that hold the wheelchair together and enable folding functionality.
- Electric components – motors, electromagnetic brakes, controller, batteries, charger.

13.2. What Is Not Covered

This warranty does not cover:

A. Damage Due to External Factors:

- Accidental damage from collisions, falls, or other impacts.
- Corrosion, rust, or degradation from exposure to moisture, humidity, chemicals, or non-compatible weather conditions (e.g., rain, snow, prolonged sun exposure).
- Environmental or external factors including:
 - Smoke, chemicals, animal waste, acid rain, metal dust
 - Natural disasters (e.g., floods, fire, earthquakes, storms)

B. Normal Wear and Tear, unless manufacturing defects are clearly identified:

- Natural material degradation over time, including:
 - Wear on tires and wheels.
 - Deterioration of grips, armrests, internal padding, and external upholstery.
 - Gradual loosening of mechanical joints.
- Deterioration from improper maintenance, such as:
 - Failure to follow the instructions outlined in this manual
 - Disregarding the safety warnings in this manual
 - Not cleaning the product or using harsh cleaning agents.
 - Failure to perform routine maintenance as outlined in the manual

C. Non-Warranty Services (Subject to Charges):

The following items and services are not covered free of charge and may incur costs:

- Replacement of consumable or wear items, including tires, seat cushions, fuses, plastic components, and lubricants.
- Routine maintenance services such as cleaning, inspections, adjustments, and lubrication.
- Repairs or servicing performed by unauthorized technicians or outside approved service centers.
- Maintenance required due to unauthorized modifications or alterations to the product.

D. Improper Use and Handling:

- Damage due to incorrect assembly or maintenance by non-authorized personnel
- Use exceeding the intended weight limit.
- Excessive strain on components due to leaning or applying improper force.
- Using the wheelchair in ways it was not designed for, such as:
 - Sitting on parts not meant for seating.
 - Using it on stairs or steep inclines.
- Improper storage, including:
 - Leaving the wheelchair outdoors for extended periods.
 - Storing in high-humidity or extreme temperature environments.
- Transporting heavy loads on the wheelchair beyond its intended use.

E. Unauthorized Modifications and Repairs:

- Unauthorized repairs, modifications or disassembly by unapproved service providers.
- Use of non-original spare parts or accessories.

- Altering structural components, such as:
 - Drilling holes, cutting, or welding any parts.
 - Adding third-party accessories that alter stability.

F. Cosmetic and Aesthetic Changes:

- Scratches, dents, or discoloration due to normal use.
- Changes in material appearance from cleaning agents, UV exposure, or aging.

G. Sales and Ownership Exclusions

- Products purchased from unauthorized sellers or outside official distribution channels.
- Second-hand purchases or those bought from third-party resellers.
- Replacement products issued under a previous warranty claim. Products transferred to another user (warranty is only valid for the original purchaser).

Warranty Claim Process:

If you experience an issue with your Helavo electric wheelchair, please follow these steps:

- 1. Contact Us:** Email our customer support team at [<https://helavo.onsitesupport.io/>] or use the contact option on our Amazon product page.
- 2. Provide Details:** Include your order number, product model, and a detailed description of the issue. Photos or videos of the defect may be requested for verification. This helps us quickly identify the issue and provide an appropriate solution to ensure your satisfaction.
- 3. Review and Resolution:** Our team will review your claim as quickly as possible, typically within a few business days. If approved, we will offer one of the following solutions:
 - Repair or replacement of the defective part.
 - Replacement of the product.

Care and Maintenance Tips:

To maximize the lifespan of your Helavo electric wheelchair, follow these recommendations:

- Regularly clean and inspect key components such as brakes, wheels, controller and armrests.
- Avoid usage on excessively rough terrain or exposure to extreme weather conditions.
- Store the wheelchair in a dry, temperature-controlled environment when not in use.

*Any replacement product or component provided under this warranty will carry the remaining duration of the original warranty, in compliance with consumer protection laws.

* This product comes with a standard 3-years warranty for the frame and 1-year warranty for the electric components that applies automatically.

14. Declaration of Conformity

Helavo GmbH declares that the H2010 electric wheelchair complies with the applicable requirements of the **European Medical Device Regulation (EU) 2017/745**.

The device meets the following harmonized standards, including but not limited to:

- EN ISO 12184: Electrically powered wheelchairs and scooters – Requirements and test methods
- ISO 7176-21:2009: Requirements and test methods for electromagnetic compatibility
- EN 60601-1-2:2015 / IEC 60601-1-2:2014: EMC testing for medical electrical equipment
- EN ISO 14971: Application of risk management to medical devices (design principle applied)

Users are encouraged to report any serious incidents or malfunctions to Helavo (**e-mail: support@helavo-mobility.com**) as part of post-market surveillance.

Thank You

Thank you for choosing our product. We hope your new electric wheelchair enhances your mobility and daily comfort.

If you have any questions, concerns, or need assistance, our customer support team is here to help. Please don't hesitate to contact us at:

support@helavo-mobility.com

Your feedback means a lot to us. If you're satisfied with your experience, we would greatly appreciate it if you left us a review—it helps others discover our products and supports us in continuing to improve.

Thank you for being part of the Helavo community!

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