Electric Wheelchair H2320

User instructions

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Mode d'emploi

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Bruksanvisningar

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User instructions

3

USER INSTRUCTIONS

35

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66

MODE D'EMPLOI

97

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128

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159

BRUKSANVISNINGAR

190

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221

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Table of contents

1. Introduction	4
2. Intended Use	4
3. Symbols Legend	5
4. Safety Warnings	6
5. Technical Parameters	12
6. Scope of delivery	13
7. Description of the electric wheelchair	14
8. Assembly & Setup	15
8.1. Folding and Unfolding the Wheelchair	16
8.2. Footrest Adjustment	17
8.3. Install the Controller	17
8.4. Install the Anti-tip wheels	18
8.5. Raising and Lowering the Armrest	18
8.6. Removing and Installing the Battery	19
8.7. Driving Modes: Electric & Manual	20
8.8. Description of Controller	21
8.9. Battery and charging information	23
8.10. Seat belt	26
8.11. Using the Wheelchair Safely: Entry, Exit, and Slope Navigation	26
9. Electric Wheelchair Range	27
10. Maintenance and Self-Check	27
11. Storage & Transportation Guidelines	29
12. Warranty Terms and Conditions	30
13. Declaration of Conformity	33

1.Introduction

Thank you for choosing the **Helavo H2320** electric wheelchair. Please read this manual carefully before use, and be sure to use the device correctly and safely. It provides important information on the wheelchair's design, features, safety instructions, and care guidelines.

Store this manual in a location where it can be referred to whenever needed.

If you have any questions regarding the assembly, the electric wheelchair or if you are unsure about anything, please contact our customer service at the e-mail address:
support@helavo-mobility.com

2.Intended use

Our powered wheelchair is for disabled or persons with reduced mobility as a walking vehicle, suitable for both indoor and outdoor environments. It is intended for use on solid ground by persons with a maximum weight of 120 kg.

The user should have the necessary physical, visual and cognitive abilities to operate the wheelchair safely.

This product combines modern design with advanced technology. Its quick disassembly makes it easy to store or transport in a car trunk, offering added convenience for travel and daily use.

Clinical Evaluation and Performance

In accordance with Annex XIV of Regulation (EU) 2017/745 (MDR), the Helavo H2320 electric wheelchair has demonstrated safe and effective performance in its intended use environment, as supported by user feedback, technical testing, and post-market surveillance data.

Company Headquarters:
Helavo GmbH
Address: Hauptstr. 46, 64342 Seeheim-Jugenheim,
Germany
Website: www.helavo.de

3. Symbols Legend

This manual and packaging use specific symbols to highlight important information. Each symbol is defined below—please pay close attention to their instructions.

WARNING: Indicates a potentially hazardous situation. Failure to follow the specified instructions may result in personal injury, equipment damage, or product malfunction.

PROHIBITED: The following actions are strictly forbidden under all circumstances. Performing them may result in personal injury and/or damage to the equipment.

Recyclable – Dispose of responsibly through local recycling systems.

It is prohibited to dispose with household waste

Manufacturer

CE-Marking

Date of manufacture

Maximum load capacity

Catalog No.

Keep Dry

Batch code


Fragile

Medical Device

Unique device identifier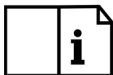

Consult Instructions for use

4. Safety Warnings

The terms “Warning” and “Attention” in this manual highlight operations that could lead to personal injury or property damage if not followed properly.

⚠ Warning! Do not operate the wheelchair without reading the manual thoroughly. Please read the following warnings carefully to avoid damage or personal injury caused by improper operation.

⚠ Warning! Wheelchair users should take appropriate safety measures and obey local pedestrian traffic rules. The company is not responsible for personal injury and/or product damage caused by improper operation. When you sit in the wheelchair, it may be difficult for others to see you. Use of safety equipment such as reflectors, reflective clothing, lights and safety signs is critical to your safety.

⚠ Warning! Please consult your health care professional regularly to ensure that there are no health or physical conditions that limit or compromise your safe operation of the wheelchair. If you are taking prescription or over-the-counter medications, or if physical activity is limited, please consult your physician. Some medications and medical conditions may impair your ability to operate the wheelchair.

⚠ Warning! All wheelchairs are not the same. Please take time to learn before driving.

⚠ Warning! Always turn off the power before entering or exiting the wheelchair.

⚠ Warning! Being able to operate a wheelchair safely needs time and practice. Before becoming comfortable with driving, practice in a safe, open area. Please drive smoothly and avoid weaving or making sharp turns.

⚠ Warning! Please do not use the wheelchair for other purposes, applications such as weight training, sports and athletics, traction, moving or dragging may cause safety hazards and excessive stress in wheelchairs.

⚠ Warning! This wheelchair is for one person only, the maximum load weight is 120kg with no passengers allowed. Do not carry anything to avoid overloading and possible rollover.

⚠ Warning! Before getting on or off the wheelchair, sit as far back as possible to prevent injury from tipping over.

⚠ Warning! The armrests are not designed to be used as transfer bars. Do not use them to transfer in or out of the chair, as this may cause damage or lead to personal injury. Also, avoid placing your full weight on the footrest, as this may cause the chair to tip over, resulting in injury.

⚠ Warning: Be extremely cautious when using oxygen near electrical circuits or combustible materials. Contact your oxygen supplier for detailed information on the safe use of oxygen.

⚠ Warning: To prevent the wheelchair from rolling or losing control, do not place it in manual mode on an inclined surface.

⚠ Warning: If you plan to remain seated in one position for an extended period, turn off the power to prevent accidental movement of the wheelchair.

⚠ Warning: Keep yourself, your clothing, and other objects away from the wheels while the wheelchair is in motion. Do not drag anything behind the wheelchair while driving. Loose clothing or items may get caught in the tires or moving parts, leading to damage or injury.

⚠ Attention: Do not connect any other devices to the electronic system or use the wheelchair battery to power external devices.

⚠ Attention: When disconnecting the wiring harness, hold the connector firmly to avoid damage. Do not pull on the wiring directly.

⚠ Warning: Keep all charger connector terminals clean and dry. Avoid exposure to moisture to prevent damage to the electrical system or risk of personal injury.

⚠ Warning: Ensure the safety of the wheelchair and battery during transportation. Never transport the wheelchair or battery with combustible materials.

⚠️ Precautions for Uphill and Downhill Driving:

To ensure safety when navigating slopes or uneven terrain, please observe the following guidelines:

- Avoid steep inclines, tilted surfaces, high steps, deep ditches, and channels. The recommended maximum slope angle is less than 6°.
- When driving on a slope, always move straight up or down—never at an angle—to prevent tipping.
- Do not drive sideways across slopes or tilt the wheelchair by climbing with one side first.
- Slow down when descending. If the wheelchair picks up speed unexpectedly, release the joystick to stop, then gently push it forward to regain controlled movement.
- Never switch to manual mode while on a slope.
- Do not attempt to climb stairs or cross high steps.
- When crossing ditches, ensure the wheels approach at a 90-degree angle for stability.

Note: When driving on a slope, releasing the joystick may cause the wheelchair to briefly roll backward. Whether moving forward or in reverse, the wheelchair may roll back up to 30 cm before the brakes fully engage.

⚠️ Warning: Always keep both front wheels in contact with the surface when climbing. Approaching a slope at an angle or unevenly may increase the risk of tipping.

⚠️ Warning: Do not attempt to climb over obstacles or cross ditches higher than 25 mm. Doing so may result in instability or injury.

⚠️ Warning: Reverse driving is only permitted on flat surfaces. Operate slowly and smoothly while reversing, and stop frequently to check for obstacles. To prevent rollover, do not reverse downhill or on a slope.

Warning: Hazardous Driving Conditions:

To avoid accidents, injury, or damage to the wheelchair, do not operate the wheelchair under the following conditions:

- On hazardous surfaces such as snow, ice, wet grass, fallen leaves, mud, sand, gravel, or uneven terrain, which may cause loss of traction or control.
- In bad weather, including rain, heavy fog, strong winds, or snow. This electric wheelchair is not waterproof. Exposure to moisture can damage the electronics and increase the risk of short-circuiting or malfunction. If the wheelchair becomes wet, dry it immediately to prevent damage.

- Near bodies of water such as streams, lakes, ponds, or the sea, and never attempt to drive through water.
- In areas with heavy pedestrian or vehicle traffic, where maneuvering could be dangerous.
- Along open edges without protective barriers, such as ditches or elevated platforms.
- Near or across railroad tracks. If crossing is absolutely necessary, proceed with extreme caution:
 1. Stop and assess the situation first.
 2. Cross tracks at a right angle.
 3. Ensure the wheels do not get caught between or on the rails.

 **PROHIBITED!** Do not expose the wheelchair to open flames or high heat sources, as this may cause fire or damage.

 **PROHIBITED!** Keep your wheelchair in a dry and clean environment. Do not use the wheelchair in the shower, bath, pool, or sauna.

 **PROHIBITED!** Do not attempt to use the wheelchair on stairs or escalators.

 **Warning:** For safe handling, always turn off the power before moving the wheelchair. Accidental contact with the joystick may cause unexpected movement. Ensure the brakes are locked to prevent the wheelchair from rolling during handling.

 **Warning:** When entering an elevator, the wheelchair may be driven inside, but the power must be turned off immediately, and the user should sit firmly to ensure the wheelchair remains stationary.

 **Warning:** Do not allow children to play with or operate the wheelchair.

 **Warning:** This wheelchair must not be used inside an MRI room when in a hospital setting.

 **Warning:** If the wheelchair requires repair or modification, please contact the customer service center. Do not attempt to modify it yourself, as this may lead to malfunctions or accidents.

⚠ Warning: Electric wheelchairs must not be transported with the brakes disengaged. We recommend placing the wheelchair in the vehicle's trunk and securing it firmly to prevent movement during transport.

⚠ Warning: Do not lean forward or bend down to pick up objects while seated in the wheelchair, as this may cause the wheelchair to tip and result in injury.

⚠ Warning: After any significant impact, fall, or crash, inspect the wheelchair thoroughly for structural or electronic damage before continued use. Contact a technician if unsure.

⚠ WARNING: Electromagnetic Interference (EMI)

Sources of radio waves—such as radio and TV stations, amateur radio transmitters, two-way radios, and mobile phones—can interfere with the control system of your power wheelchair.

To reduce the risk of unexpected movement or braking, which may result in serious injury, please follow these precautions:

1. Do not use portable communication devices (e.g., CB radios, mobile phones) while the wheelchair is powered on.
2. Avoid close proximity to strong transmitters, such as broadcasting towers or other high-power radio sources.
3. If the wheelchair behaves unexpectedly or releases its brake without warning, immediately switch off the power if it is safe to do so.
4. Do not modify or add unauthorized accessories to the wheelchair, as this may reduce its resistance to electromagnetic interference.
5. Report any incidents of unexpected movement or brake failure to the manufacturer. Always remain alert to nearby radio sources and follow all safety advisories.

This symbol indicates that, in accordance with local laws and regulations, this product must not be disposed of with household waste. When the wheelchair reaches the end of its useful life, please take it to a designated collection point for the recycling of electrical and electronic equipment. Proper disposal helps conserve natural resources and ensures that the product is recycled in a manner that protects human health and the environment.

Important Disposal Guidelines:

- **Electrical Components:** The wheelchair contains electrical and electronic parts that fall under the scope of the WEEE Directive. These components must be disposed of through authorized recycling facilities equipped to handle electronic waste.
- **Batteries:** If the wheelchair includes lithium-ion batteries, they must be removed and recycled separately in accordance with local regulations governing battery disposal.
- **Plastic and Metal Parts:** Non-electronic components, such as plastic covers, the vehicle frame, and pipe structures, should be taken to professional recycling centers that can process these materials appropriately.

Contraindications and Residual Risks

This wheelchair is not recommended for users who:

- Have recently undergone surgery affecting balance or movement
- Suffer from severe cognitive impairments or disorientation
- Have significant uncorrected visual impairments
- Require the use of an intravenous pole, oxygen tanks (unless used per guidelines), or other attachments without proper integration
- Weigh more than 120 kg
- Experience frequent loss of consciousness or seizures

Residual Risks remain even with correct usage, including:

Residual risk that cannot be fully eliminated	Practical mitigation / user action
Tipping on steep or uneven terrain	• Keep to gradients $\leq 6^\circ$ and drive straight up/down slopes, never sideways or diagonally • Always deploy the supplied anti-tip wheels; never remove them • Reduce speed when descending; avoid high kerbs, stairs and loose surfaces
Reduced responsiveness due to electromagnetic interference (EMI)	• Do not use CB radios, mobile phones, or other transmitters while the chair is powered on • Stay clear of strong broadcast towers; switch off the chair immediately if it behaves unpredictably
Battery overheating if stored or used improperly	• Keep batteries away from direct heat sources and open flames; store/charge at 0 – 40 °C • Follow the charging schedule and disconnect the charger once the green LED is lit

Skin injury from prolonged seating without repositioning	• Shift weight (or perform a brief lift) at least every 30 min • Follow clinical advice on sitting time limits and schedule regular pressure-area checks with a healthcare professional
--	--

5. Technical Parameters

Model	H2320
Body Frame	Magnesium Alloy Material
Maximum Load Capacity	120 kg
Length L (mm)	1030
Width B (mm)	605
Height H (mm)	940
Folded size	790 x 605 x 360 mm
Seat height	530 mm
Seat width	430 mm
Effective seat width	480 mm
Backrest height	430 mm
Net weight (without battery)	17.3 kg
Battery Weight	1.86 kg
Maximum Speed forward	Up to 6 km/h
Maximum Speed backward	Less than 3 km/h (0.8 m/s)
Horizontal road braking	≤ 1200 mm
Climbing Ability	6°
Maximum obstacle climbing	25mm
Min Turning Radius	≤ 900 mm
Battery capacity	Lithium Battery 24V/12Ah, UN38.3 certified
Theoretical Driving Distance	≥ 16 km
Brake system	Automatic electromagnetic brake system
Motor	Brushless DC motor; 24V, 250W, 2pcs
Maximum Output Current of Controller	35A
Maximum Output Current of Charger	2A
Front Wheel Specification	Solid Tyre, 175mm
Rear Wheel Specifications	Solid Tyre, 270mm

- Note:**
1. Dimensional tolerance: ± 50 mm; angle tolerance: $\pm 3^\circ$; weight tolerance: ± 2 kg.
 2. Weight capacity is tested using a dummy or person of equivalent weight.
 3. The maximum range is measured on a flat surface at an average speed and may vary depending on terrain, user habits, and other conditions.

6. Scope of delivery


The Helavo H2320 comes with the following parts:

1. Electric wheelchair

2. Lithium battery 24V/12Ah

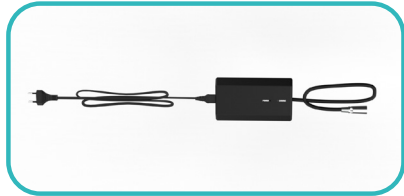3. Charger

4. Controller and Knob Screw

5. Anti-tip wheels, 2 pcs


6. Seat and backrest cushions


7. Rear Bag

8. Side Bag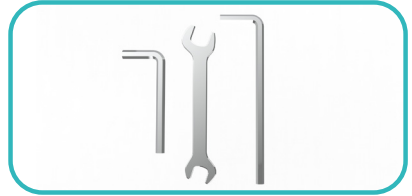

9. Tool Kit

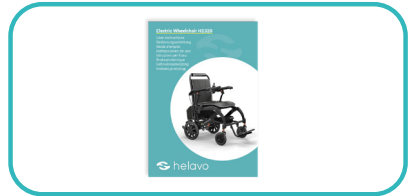10. User instructions

11. Product inserts

7. Description of the electric wheelchair

Fig. 1

Fig. 2

Normal Operating Conditions:

The electric wheelchair is designed to function properly under the following environmental conditions:

- Ambient Temperature: 5°C to 40°C
- Relative Humidity: $\leq 80\%$
- Atmospheric Pressure: 86 kPa to 106 kPa
- Input Voltage (Charger): AC 110V-220V, 50-60Hz
- Output Voltage (Charger): DC 24V

8. Assembly & Setup

Installation Hint:

- No tools or additional equipment are required to assemble the electric wheelchair.
- Before opening the box, inspect the carton for any signs of damage or crushing.
- Read all provided documentation thoroughly to become familiar with the product's assembly and operation procedures.
- Unpack the contents carefully, removing the main body of the wheelchair, seat cushion, user manual, and any other included materials.
- Gently remove all protective materials, such as film, plastic, and foam, used to safeguard the components during shipping. Take care not to scratch or damage any parts during this process.

Please keep this **packaging and protective inserts** for the entire warranty period; they're designed to safeguard the wheelchair in the event of service or warranty return.

8.1. Folding and Unfolding the Wheelchair:

A. Unfolding:

When you first receive the wheelchair, it will be folded inside the box. To unfold it, place the wheelchair flat on the ground with the wheels facing downward.

Hold the top of the backrest with one hand and the front edge of the seat with the other. Gently pull them apart until the wheelchair is fully unfolded—**you should hear a click when it locks into place.**

Fig. 3

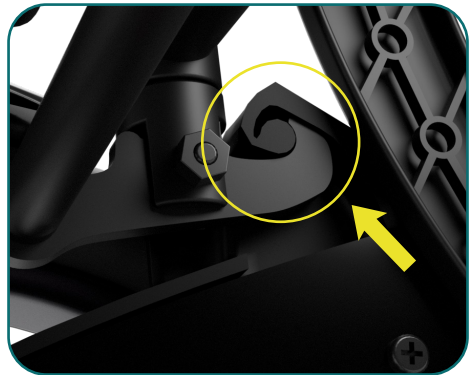
Fig. 4

⚠ Ensure that the latches located beneath the seat at the back are securely locked in place (Fig 4).

B. Folding:

To fold the electric wheelchair, lift the rod located beneath the backrest (see Figure 5) and push the backrest down toward the seat in the direction indicated by the arrow in Figure 6.

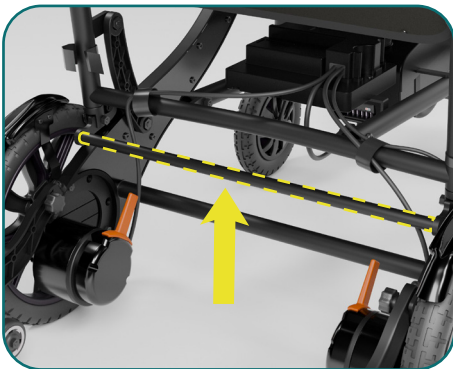
Fig. 5

Fig. 6

8.2. Footrest Adjustment

Lower the footrest from its vertical position for proper support during use.

Fig.7

8.3. Install the Controller.

Loosen the knob located beneath the right armrest, insert the controller tube into the armrest mounting tube, adjust it to the desired position, and then securely tighten the knob:

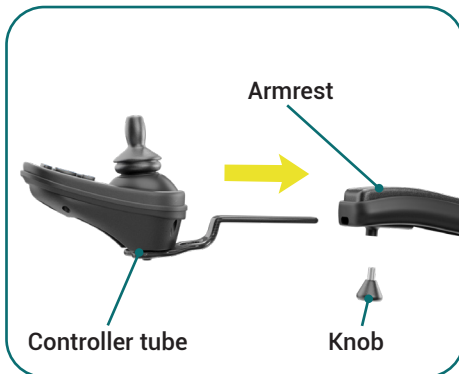
Fig.8

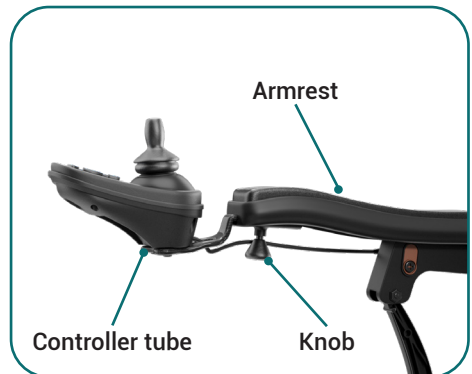
Fig.9

Note: The controller can be mounted on either the left or right armrest, depending on the user's preference.

8.4. Install the Anti-tip wheels

First, remove any protective packaging. Then, rotate the anti-tip support until it is positioned perpendicular to the rear wheels and secure it in place using the supplied knob screw.

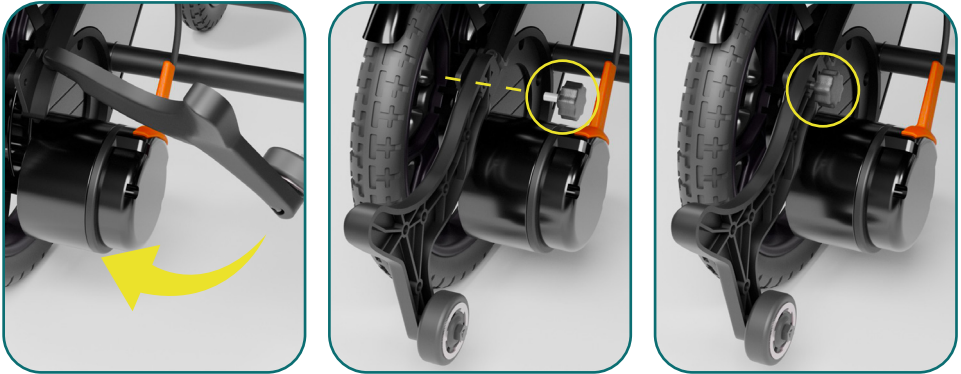
Fig. 10

Repeat the process for the second anti-tip wheel.

⚠ Warning: Always use both anti-tip wheels. Operating without them may lead to tipping and serious injury.

8.5. Raising and Lowering the Armrest

To lift the armrest, press the release button while holding the armrest tube, then raise it upward (See Figures 11 and 12). This allows easier access when getting on or off the wheelchair.

Before driving, always lower the armrest back into place and ensure it is securely locked.

Fig. 11

Fig. 12

8.6. Removing and Installing the Battery

The battery is pre-installed beneath the seat. To activate it, press the power button located directly on the battery (that side of the button marked with "I" (ON).)

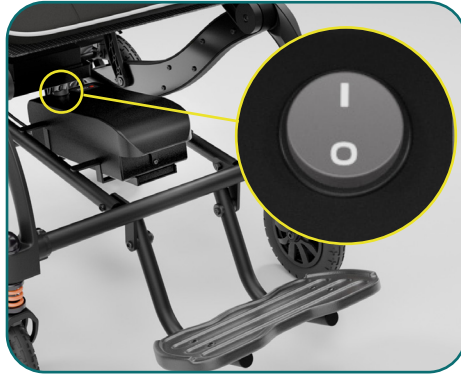
Fig.13

To remove the battery:

Lift the battery latch upward, then slide the battery case forward to detach and remove

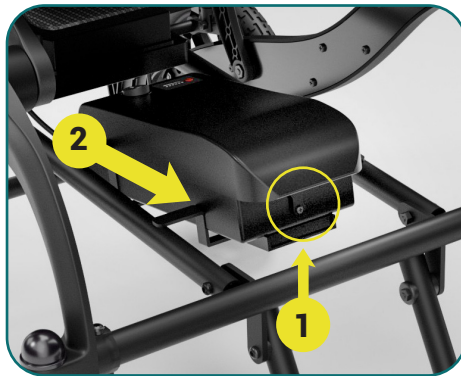
Fig.14

To re-install the battery:

Slide the battery case along the rail until it reaches the bottom. The latch will automatically lock it into place. A click sound will confirm that the battery is securely locked in place.

8.7. Driving Modes: Electric & Manual

This electric wheelchair features two driving modes:

- **Electric Mode** – The wheelchair is powered and controlled using the joystick. In this mode, it cannot be pushed manually.
- **Manual Mode** – The wheelchair can be pushed by an assistant. The joystick is disabled and does not control movement.

⚠ Warning: Never switch to Manual Mode on an incline or slope, as the wheelchair may roll uncontrolled, leading to injury.

To switch between modes, use the colored levers located at the lower rear of the wheelchair:

- Levers **down**--Electric Mode
- Levers **up**--Manual Mode

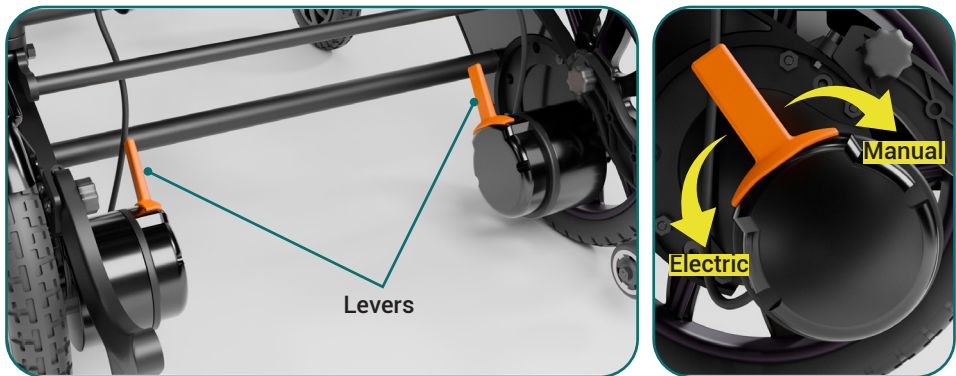
Fig. 15

⚠ Attention: Before switching between Manual and Electric Mode (by moving the levers up or down), always make sure to turn off the controller first.

Note: Electric Mode Also Acts as a Brake

When the wheelchair is in Electric Mode (levers down), the motors remain engaged and act as an automatic brake, preventing the wheelchair from rolling.

If you're using the wheelchair in Manual Mode (levers up) and need to park it or leave it stationary for an extended period, it is recommended to switch back to Electric Mode to activate the braking function and keep the wheelchair securely in place.

8.8. Description of Controller

The joystick unit integrates all essential electronic controls needed to operate the wheelchair. It features the following components:

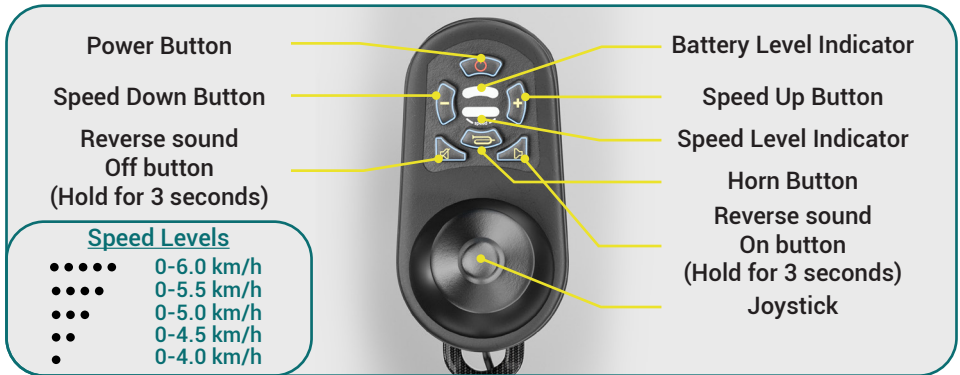
Fig. 16

⚠ Attention: Before turning on the controller, ensure that the joystick is in the center (neutral) position. Starting the chair with an off-center joystick can cause sudden, unintended movement.

- **To turn on the controller**

Press the Power button. The battery and speed indicator lights will turn on, and a beep from the buzzer will confirm that the controller has powered on successfully.

- **To turn off the controller**

Press the Power button again. All indicator lights will turn off, and the controller will cease power output.


⚠ In case of an emergency, you can press the Power button at any time to immediately shut off the system.

- **Adjusting the Speed**

You can adjust the maximum driving speed of the electric wheelchair according to your personal preferences and environment. There are five speed levels available (please check Fig. 16). After powering on, the **Speed Level Indicator** displays the current maximum speed level using a series of bars.

-Press the **Speed Down Button** to decrease the speed by one level.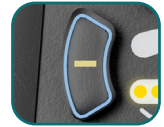

-Press the **Speed Up Button** to increase the speed by one level. Each press adjusts the number of illuminated bars, which represent the current speed setting.

- **Horn Activation**

To use the horn, press and hold the horn button. The horn will sound continuously while the button is pressed and will stop once released.


- **Activating or Deactivating the Rear Sound Alert**

-**To disable the Reverse Sound Alert**, press and hold this button until a confirmation beep is heard.

-**To turn the Reverse Sound back on**, press and hold this button until a confirmation beep is heard.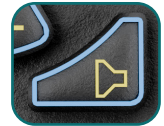

- **Battery Level Indicator**

The battery level indicator illuminates when the wheelchair is powered on, showing the estimated remaining battery charge.

-**All LEDs on (Green)**: Battery is fully charged.

-**Two or three LEDs on (Orange)**: Battery level is low. Recharge soon—especially before longer trips.

-**One LED on (Red)**: Battery is critically low. Recharge immediately to avoid interruption.

- **Joystick Operation**

The joystick controls both the direction and speed of the electric wheelchair. The further you push the joystick in any direction, the faster the wheelchair will move in that direction.

- Push **forward** - move forward
- Pull **backward** - move in reverse
- Push **left** - turn left on the spot
- Push **right** - turn right on the spot

The joystick provides **smooth 360° control**, allowing the wheelchair to move in any direction intuitively—based on both the angle and pressure applied.

When the joystick is released, it automatically returns to the center position and activates the **braking system**, bringing the wheelchair to a safe stop.

Controller Beep Code Diagnostic Table

When a fault is detected, the controller will emit a sequence of audible beeps. The number of beeps corresponds to a specific issue. Use the table below to identify the fault and follow the recommended actions. If the issue persists after completing the suggested checks, please contact our customer service team for further assistance.

Beep Count	Fault Description	Recommended Action
2+1 beeps	Communication failure	Check the internal connections inside the joystick
2+2 beeps	Right motor fault	Check the right motor and wiring connection
2+3 beeps	Right brake fault or poor cable connection	Ensure the right brake lever is in place and not damaged
2+4 beeps	Left motor fault	Check the right motor and wiring connection
2+5 beeps	Left brake fault or poor cable connection	Ensure the left brake lever is in place and not damaged
2+6 beeps	Overcurrent detected	Check for motor blockage, short circuit or possible joystick damage
2+7 beeps	Joystick lever error	Ensure the joystick moves freely and its internal connections are intact
2+8 beeps	Joystick malfunction	Contact customer service for inspection and repair
2+9 beeps	Battery fault	Charge the battery fully. If the problem persists, replace it with a compatible model.

8.9. Battery and charging information:

The charger has a built-in indicator light with two status modes:

- 1.Blue** – Charging in progress
- 2.Green** – The battery is basically fully charged, but please keep charging for 1-2 hours. We recommend charging the batteries for 6-8 hours. Remove the power plug and charger when the battery is fully charged.

Initial Use of New Batteries:

To optimize battery performance and longevity:

- Before first use**, fully charge the new battery. This will bring it to approximately 90% of its maximum capacity.

- After 4–5 full charge/discharge cycles**, the battery will gradually reach 100% of its full capacity, while also helping to extend its overall lifespan.

Charging Methods:

Method 1 - Charging through the controller

⚠ Attention! Before charging, ensure that the wheelchair controller is turned off.

Plug the metal 3-pin connector of the charger into the 3-hole charging port on the controller. Then, insert the other end of the charger into a wall power outlet.

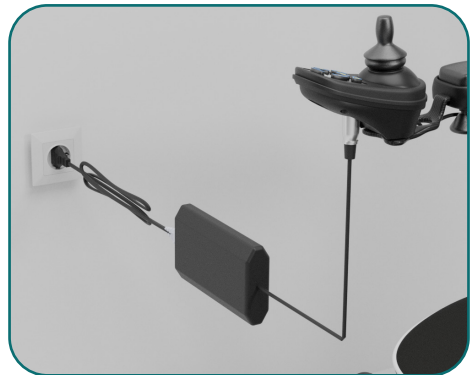
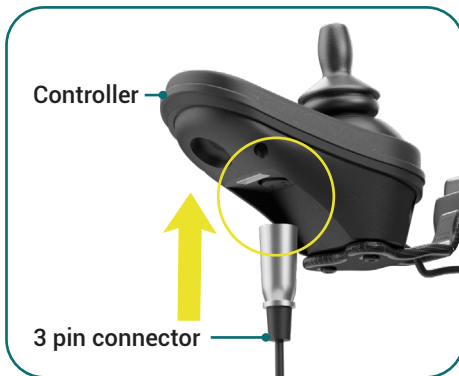
Fig. 17

Method 2 - Charging the battery separately

Remove the battery from the wheelchair and open the circular charging port cover at the bottom. Insert the charger plug into the battery port and connect the other end to a power source.

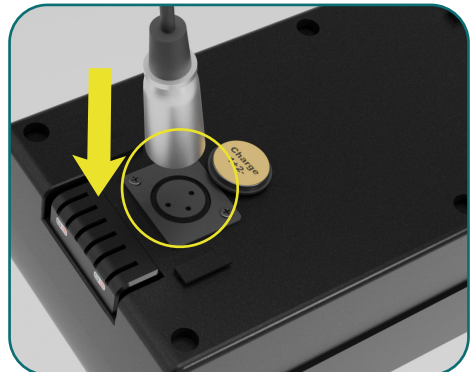
Fig. 18

⚠ Charging Guidelines and Safety Notes:

⚠ Warning: Always charge the battery in a well-ventilated area, away from flammable materials, heat sources, or direct sunlight.

Charging in enclosed or poorly ventilated spaces may lead to heat buildup and increase the risk of fire or explosion. Never charge the wheelchair near curtains, bedding, or other flammable items.

⚠ Warning: Keep children and pets away from the wheelchair during charging. Unsupervised contact during charging poses electrical, fire, and injury risks.

- To ensure safety and maintain your product warranty, only use the original charger provided with your electric wheelchair. Using unauthorized chargers may: Damage the battery or wheelchair, Void your warranty, Increase the risk of fire or serious malfunction.

- Do not leave the charger connected to both the power source and the batteries for more than 24 hours. Extended connection beyond 24 hours is unnecessary and may affect battery performance over time.

- Avoid disconnecting the battery before it is fully charged (indicated by the green LED on the charger), as doing so may reduce battery lifespan.

- After charging, always disconnect the charger from both the power outlet and the controller to avoid unintended power drain.

- If the wheelchair is not in use for extended periods, recharge the batteries at least once a month to preserve battery health and prevent capacity loss.

- The electric wheelchair is equipped with a safety circuit to prevent movement during charging.

Charge the battery regularly, even when the wheelchair is not in use, to prevent deep discharge and extend battery life.

- If storing the wheelchair for over 5 days, charge the battery for 6 hours, unplug the charger, and ensure the battery is disconnected.

- Inspect batteries regularly for visible damage. Contact the customer service immediately if any issues are found.

Important: Avoid short-circuiting battery terminals. Remove all metal jewellery (e.g., watches, necklaces) before inspecting the battery.

⚠ Battery Handling and Safety Warnings

- Handle and dispose of batteries and chargers with care.

- Do not incinerate batteries. Dispose of them according to local environmental regulations.

- Keep batteries away from heat sources. Overheating may lead to an explosion.

- Do not puncture, crush, or expose the battery to high external pressure. These actions may cause internal short-circuits or overheating.

- Avoid charging in environments with open flames or while smoking.

- Do not touch the metal battery terminals.

- Use only approved batteries. If the battery casing is damaged or leaking, do not attempt to repair it—contact our customer service immediately.

- If handling a leaking or damaged battery:

- Wear insulated gloves and safety goggles

- Replace the battery immediately and dispose of it following proper guidelines

⚠ In Case of Battery Damage or Leaks

Battery fluid is corrosive:

- Avoid contact with skin or clothing – rinse immediately with soap and water.

- If it contacts the eyes, flush with clean water for at least 10 minutes and seek medical attention.

- Contact our customer service for safe removal and replacement.

8.10. Seat belt

For your safety, always wear the seat belt while operating the wheelchair. The seat belt helps prevent accidental falls or injury, especially when driving over uneven terrain or during sudden stops. Ensure it is securely fastened before each use.

8.11. Using the Wheelchair Safely: Entry, Exit, and Slope Navigation

Getting Into the Wheelchair:

- Make sure the **power is turned off** before getting seated.
- Carefully transfer your body into the seat. You may use a crutch or support to assist with the movement.
- Place your feet securely on the footrests.
- Press the **power button** to turn the wheelchair on. Use the **joystick** to control forward, backward, left, and right movement.

Getting Out of the Wheelchair:

- Ensure the wheelchair has completely stopped.

- Press the power button to turn off the device.
- With the help of a crutch or support, slowly transfer yourself out of the seat.

Safety Precautions: Slopes and Uneven Surfaces

Always approach slopes head-on (never sideways) and drive slowly. Avoid steep or uneven surfaces, stairs, high curbs, or deep ditches. Never switch to manual mode while on a slope.

9. Electric Wheelchair Range

The provided Lithium 12Ah 24V battery has a range of 16km.

Please be aware that the driving range of an electric wheelchair can vary depending on multiple factors.

Although we follow standardized testing procedures to estimate range, real-world results can vary due to differences in motor power, total weight, and environmental conditions.

Factors That May Reduce Driving Range:

- User weight exceeds 120 kg
- Battery is worn out, improperly stored, or past its optimal use date
- Challenging or uneven terrain (e.g. slopes, mud, gravel, grass, snow, or ice)
- Frequent use on inclines or long uphill routes
- Extreme temperatures (hot or cold)
- Worn tires or high rolling resistance (e.g. thick carpets)
- Frequent starting, stopping, or sharp turns

Note: If you're driving outdoors and notice that the battery indicator light has turned red, you can help extend your remaining range by lowering the maximum speed setting.

10. Maintenance and Self-Check

Regular maintenance is essential to ensure the long-term performance and safety of your electric wheelchair. Basic tasks such as cleaning, checking tires, and battery care can be performed by the user. More complex servicing should be carried out by qualified service personnel.

Cleaning and Maintenance

Wheelchair Frame:

- Clean the wheelchair regularly using a soft cloth to prevent dust and dirt buildup.

Tires:

- Check tires and wear frequently.
- If the tread depth is less than 1 mm, replace the tires.

Battery:

- Charge the battery frequently to maintain optimal performance.
- To prolong battery life, avoid waiting until the battery is fully drained before recharging.

Routine Maintenance and Inspection

- Inspect all connections to ensure they are tight and secure.
- Check all cable connections, including the charger cable, to confirm they are properly attached and free from corrosion. Make sure the battery pack is correctly inserted into the slide rail.
- After each use, switch off the power to prevent unnecessary battery drain.
- The wheelchair is equipped with an automatic power-saving feature. If no activity is detected for 20 minutes, the controller will shut off automatically. To resume operation, simply turn the controller back on.
- All wheel bearings are pre-lubricated and sealed—no additional lubrication is required.
- To maintain optimal performance, inspect the wheelchair before each use and follow a regular maintenance schedule (weekly, monthly, and semi-annually). Refer to Table 2 for detailed inspection items and intervals.

Inspection Items	At any time	Weekly	Monthly	Semi- annual
Each part			●	
Turning, driving, settings, joystick operation		●		
Brakes	●			
Connections		●		
Battery Condition	●			
Tire Condition			●	
Motor				●
Cleanliness	●			

Important Note:

Do not attempt to open or modify any part of the wheelchair's electrical system, including the battery box, wiring harness, connectors, or charger. These components are not user-serviceable and should only be inspected or repaired by authorized service personnel.

11. Storage & Transportation Guidelines

Storage: If you plan to store your powered wheelchair for more than one week, fully charge the battery for up to 12 hours and then disconnect it to help prevent battery discharge during storage.

Transport: The wheelchair is designed for quick disassembly and easy transport. Follow these steps to ensure safe handling:

- Turn off the power and ensure the brake levers are engaged (locked position).
- Disconnect the Controller from the main frame.
- Remove the seat.
- Lift the wheelchair frame into the vehicle trunk.
- Store the seat separately in a secure and protected position.

To reassemble the wheelchair after transport, simply reverse the above steps.

Additional Transport Notice:

While basic transport can be done using minimal securing methods, for maximum protection during shipping or long-distance travel, we recommend using the original packaging box and securing the wheelchair inside the transport vehicle.

Note: The battery complies with international safety standards and is generally approved for transport by air, train, and ship. The 12 Ah, 24V lithium battery is 288 Wh; tested per UN 38.3; complies with IATA PI 967 Section II.

Please check with the transport provider in advance for specific requirements for lithium battery transport.

Storage & Transport Conditions

- Keep the wheelchair away from moisture, direct sunlight, and heat sources.
- Do not store outdoors or in damp environments to avoid damage to electrical components.
- Protect the unit from rain or humidity during both storage and transportation.

Environment temperature 0~+40

Relative humidity≤80%

Air Pressure 86kPa~106kPa

12. Warranty Terms and Conditions

At Helavo, we are committed to delivering top-quality mobility solutions and exceptional customer service. Our warranty policy is designed to ensure transparency and provide you with the support you need. Please review the following terms carefully.

12.1. Warranty Coverage

We offer a **3-year limited** warranty for the wheelchair frame and **1-year limited warranty** for the electric components (motors, controller, electromagnetic brakes, charger, battery) starting from the date of delivery for the first purchaser. This warranty applies only to the originally purchased product and does not cover any replacement units provided under a warranty claim. It covers essential structural components of the electric wheelchair, covering material and workmanship defects that existed at the time of delivery. It runs concurrently with statutory warranty rights and is intended to give you additional peace of mind regarding the durability and reliability of your product.

Essential structural components covered by this warranty include:

- The frame – The central supporting structure that provides stability and durability.
- Joints and hinges – Mechanical parts that hold the wheelchair together and enable folding functionality.
- Electric components – motors, electromagnetic brakes, controller, batteries, charger.

12.2. What Is Not Covered

This warranty does **not** cover:

A. Damage Due to External Factors

- Accidental damage from collisions, falls, or other impacts.
- Corrosion, rust, or degradation from exposure to moisture, humidity, chemicals, or non-compatible weather conditions (e.g., rain, snow, prolonged sun exposure).
- Environmental or external factors including:
 - Smoke, chemicals, animal waste, acid rain, metal dust
 - Natural disasters (e.g., floods, fire, earthquakes, storms)

B. Normal Wear and Tear, unless manufacturing defects are clearly identified.

- **Natural material degradation** over time, including:
 - Wear on tires and wheels.
 - Deterioration of grips, armrests, internal padding, and external upholstery.
 - Gradual loosening of mechanical joints.
- **Deterioration from improper maintenance**, such as:
 - Failure to follow the instructions outlined in this manual
 - Disregarding the safety warnings in this manual
 - Not cleaning the product or using harsh cleaning agents.
 - Failure to perform routine maintenance as outlined in the

manual

C. Non-Warranty Services (Subject to Charges)

The following items and services are not covered free of charge and may incur costs:

- Replacement of consumable or wear items, including tires, seat cushions, fuses, plastic components, and lubricants.
- Routine maintenance services such as cleaning, inspections, adjustments, and lubrication.
- Repairs or servicing performed by unauthorized technicians or outside approved service centers.
- Maintenance required due to unauthorized modifications or alterations to the product.

D. Improper Use and Handling

- Damage due to incorrect assembly or maintenance by non-authorized personnel
- Use exceeding the intended weight limit.
- Excessive strain on components due to leaning or applying improper force.
- Using the wheelchair in ways it was not designed for, such as:
 - Sitting on parts not meant for seating.
 - Using it on stairs or steep inclines.
- Improper storage, including:
 - Leaving the wheelchair outdoors for extended periods.
 - Storing in high-humidity or extreme temperature environments.
- Transporting heavy loads on the wheelchair beyond its intended use.

E. Unauthorized Modifications and Repairs

- Unauthorized repairs, modifications or disassembly by unapproved service providers.
- Use of non-original spare parts or accessories.
- Altering structural components, such as:
 - Drilling holes, cutting, or welding any parts.
 - Adding third-party accessories that alter stability.

F. Cosmetic and Aesthetic Changes

- Scratches, dents, or discoloration due to normal use.
- Changes in material appearance from cleaning agents, UV exposure, or aging.

G. Sales and Ownership Exclusions

- Products purchased from unauthorized sellers or outside official distribution channels.
- Second-hand purchases or those bought from third-party resellers
- Replacement products issued under a previous warranty claim.
- Products transferred to another user (warranty is only valid for the original purchaser).

12.3. Warranty Claim Process

If you experience an issue with your Helavo electric wheelchair, please follow these steps:

- 1. Contact Us:** Email our customer support team at [<https://helavo.onsitesupport.io/>] or use the contact option on our Amazon product page.
- 2. Provide Details:** Include your order number, product model, and a detailed description of the issue. Photos or videos of the defect may be requested for verification. This helps us quickly identify the issue and provide an appropriate solution to ensure your satisfaction.
- 3. Review and Resolution:** Our team will review your claim as quickly as possible, typically within a few business days. If approved, we will offer one of the following solutions:
 - Repair or replacement of the defective part.
 - Replacement of the product.

Care and Maintenance Tips:

To maximize the lifespan of your Helavo electric wheelchair, follow these recommendations:

- Regularly clean and inspect key components such as brakes, wheels, controller and armrests.
- Avoid usage on excessively rough terrain or exposure to extreme weather conditions.

- Store the wheelchair in a dry, temperature-controlled environment when not in use.

***Any replacement product or component provided under this warranty will carry the remaining duration of the original warranty, in compliance with consumer protection laws.**

*** This product comes with a standard 3-years warranty for the frame and 1-year warranty for the electric components that applies automatically.**

13. Declaration of Conformity

Helavo GmbH declares that the H2320 electric wheelchair complies with the applicable requirements of the **European Medical Device Regulation (EU) 2017/745**.

The device meets the following harmonized standards, including but not limited to:

- **EN ISO 12184:** Electrically powered wheelchairs and scooters – Requirements and test methods
- **ISO 7176-21:2009:** Requirements and test methods for electromagnetic compatibility
- **EN 60601-1-2:2015 / IEC 60601-1-2:2014:** EMC testing for medical electrical equipment
- **EN ISO 14971:** Application of risk management to medical devices (design principle applied)

Users are encouraged to report any serious incidents or malfunctions to Helavo (**e-mail: support@helavo-mobility.com**) as part of post-market surveillance.

Thank You

Thank you for choosing our product. We hope your new electric wheelchair enhances your mobility and daily comfort.

If you have any questions, concerns, or need assistance, our customer support team is here to help. Please don't hesitate to contact us at:

support@helavo-mobility.com

Your feedback means a lot to us. If you're satisfied with your experience, we would greatly appreciate it if you left us a review—it helps others discover our products and supports us in continuing to improve.

Thank you for being part of the Helavo community!

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